



BrainLink Pro Quick Start Manual

Tutorial Application Download


Please use your smartphone to scan the QR code below to download the **BrainLink** app, you can use the in-app instruction to master the basic use of BrianLink



Product Charging

After receiving the product, please charge the product before use. When charging you need to use the charging cable inside the package to charge. The red LED light on the module will stay on while charging, and the LED light will go off when it is fully charged. (check the picture below)

A red light is on when charging.
Red light is off when battery is fully charged.

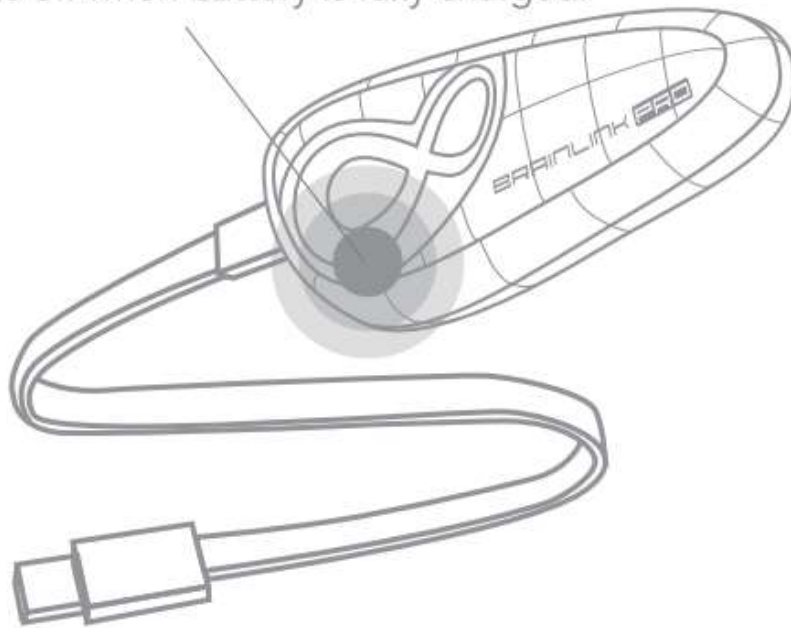

Use 5.0V--1.0A USB power source to charge.


Power Switch

Long press the button of LOGO of the core module to turn it on.

It will be accompanied by two vibrations; Short press this switch to turn it off at the same time will be accompanied by a vibration.




Assembly

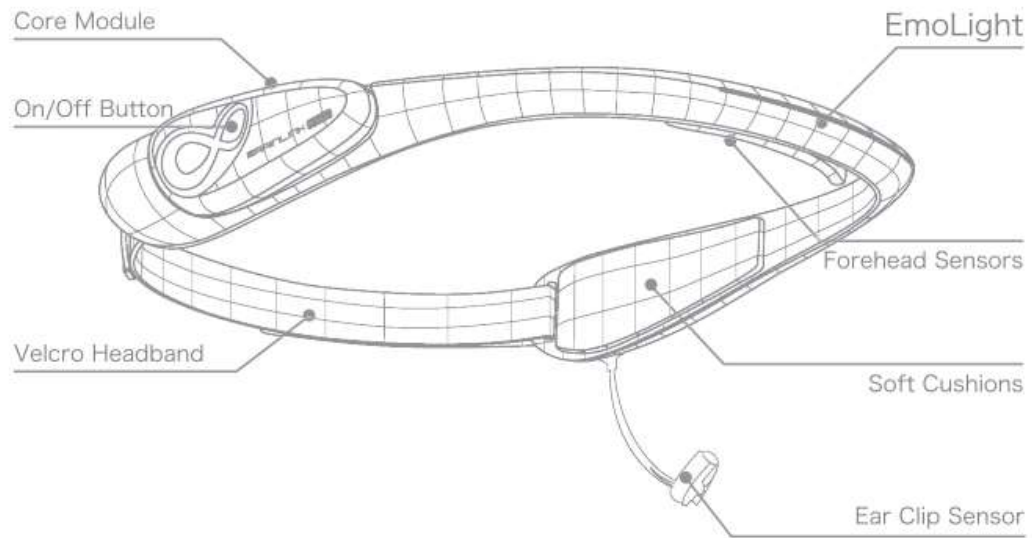


👁 Top View

1. Insert the Module from a 30 degree angle.

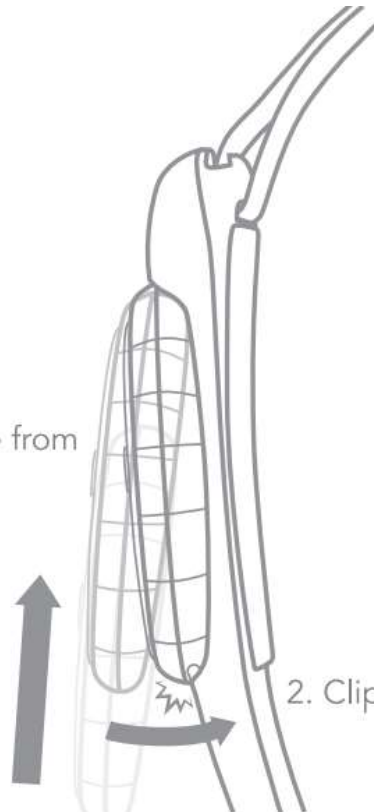

2. Clip the back of the Module to the joint.

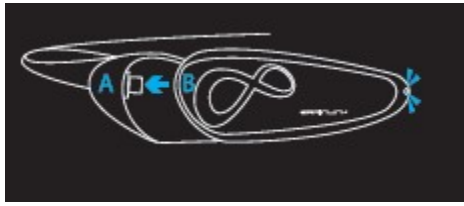


Wearing

The product comes with two headbands-the soft headband contains additional heart rate and frontal temperature sensors, which are comfortable to wear and suitable for long time wearing; the hard headband integrates EmoLight band and ear clip electrode, which can change color according to the brain state and provide more sensitive EEG data, suitable for short time high sensitivity EEG data acquisition.

Attach the fully charged core module to the base of the headband. Adjust the position of the Flexi band at the back of the headband to fit your head circumference.


Headband

2. Put on BrainLink Pro from top.

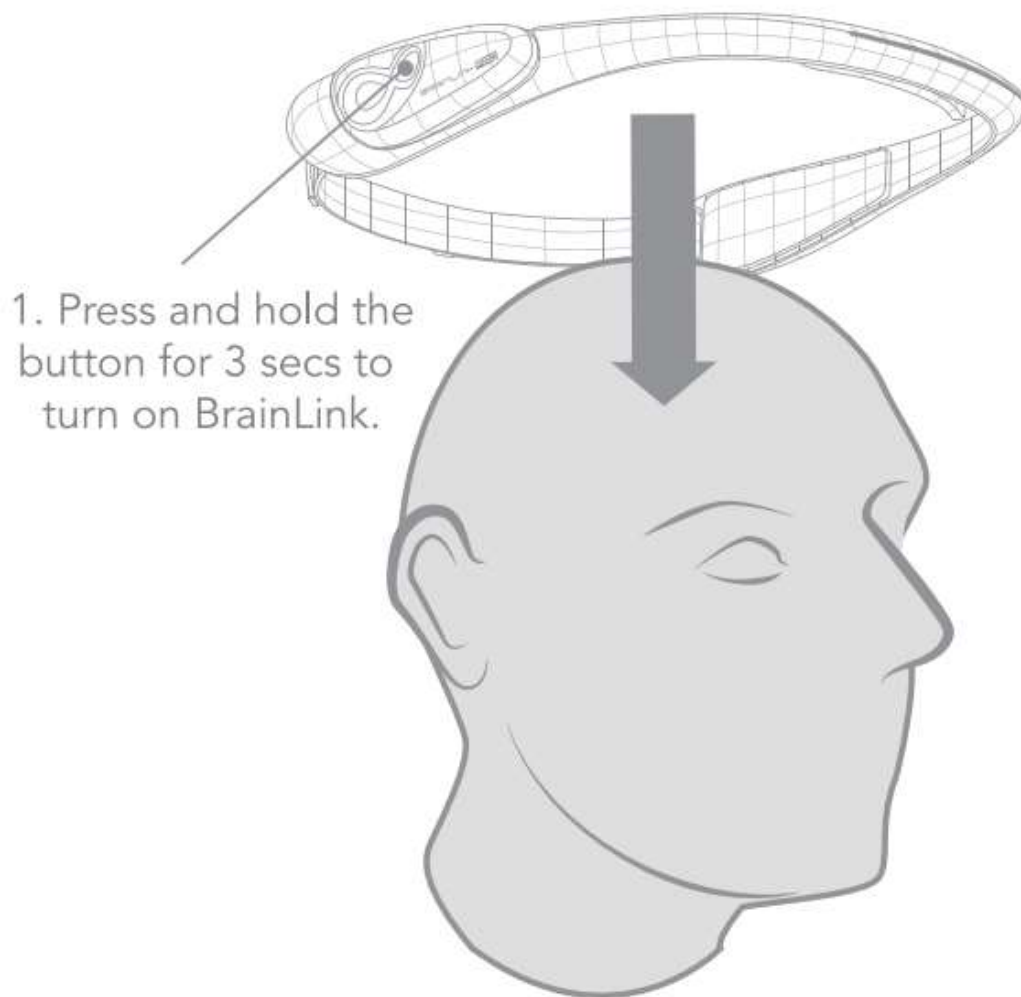


The core module of the soft headband is located above the **left ear** and the **ear clip electrode of the hard headband is clipped to the left earlobe**. There will be two vibrations to indicate the completion of wearing after about 1S of correct wearing

Place it 2-3cm above the eyebrows.
Make sure both sensors are touching the forehead skin firmly.
Avoid any hair blocking the sensors.

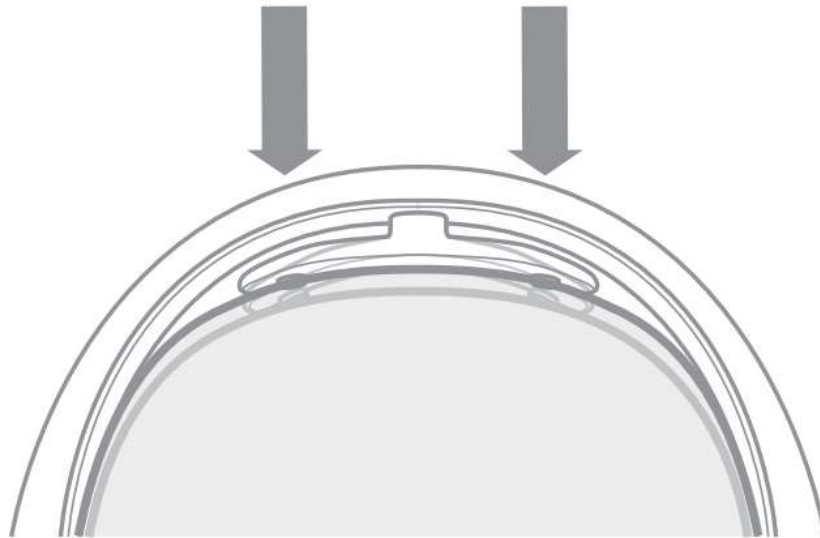


1. Adjust the Headband tightness.
Fasten it with the headband velcro.

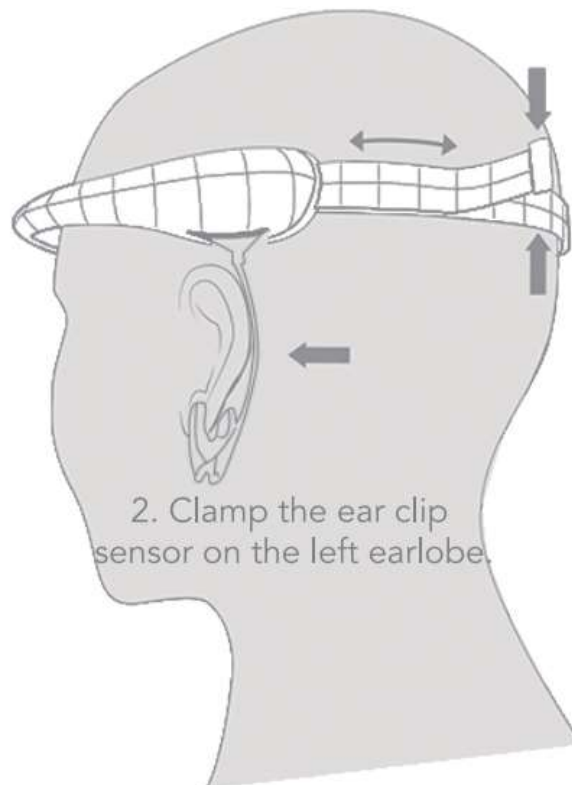

2. Clamp the ear clip
sensor on the left earlobe.

Soft Headband wearing (picture below)

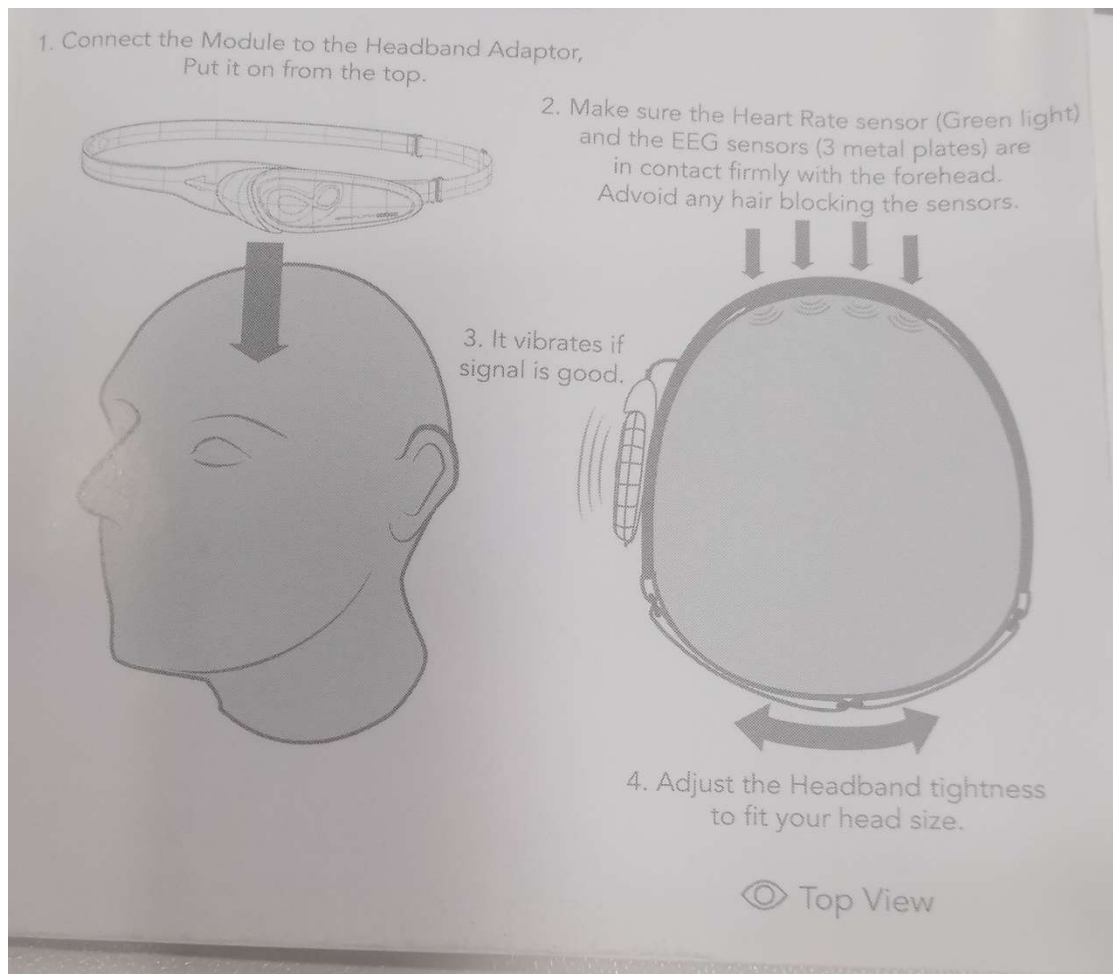

Connect App

Turn on any BrainLink APP, then turn on the power of the core module

& wait a moment ,in the APP main interface.there will be an icon in the upper right corner to show the device wearing and

connection status. When there is a LARGE dot in the middle, it means the device is wearing correctly 📶

;When there are 1-3 little dots in the middle, it means the device is not wearing or the

electrode contact is not good 📶; When there is red cross in the middle, it

means the device is not connected to Bluetooth. 📶

APP Download

You can scan the QR code below or click this link ***<https://api.macrotellect.com/api/appListView?language=en>*** to go to the official application download page to download other apps.

