




Package Items List:

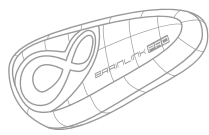


BrainLink Pro Module

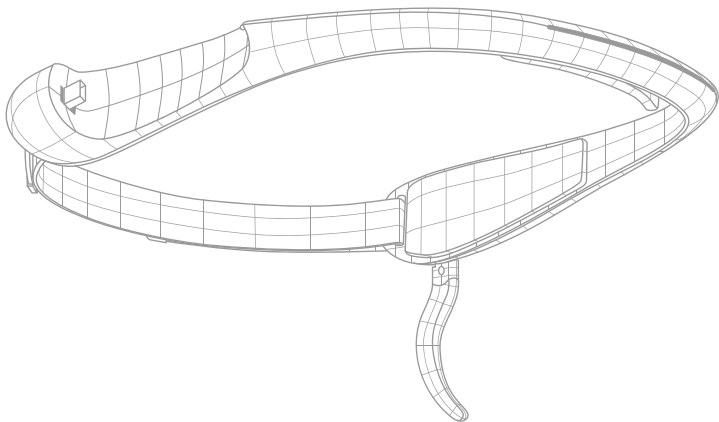

Electrode Adaptor

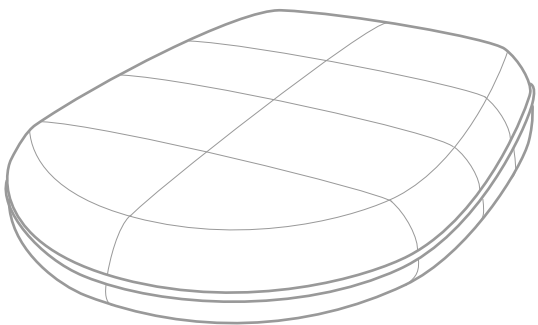

Carrying Case

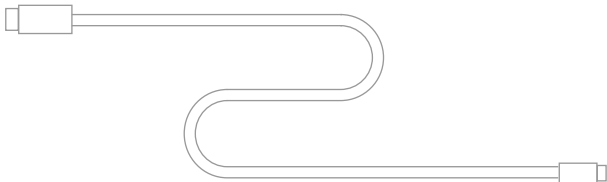


Charging Cable
(Non-standard, please keep it in safe place)



Telekinesis App AR Card & Quick Start Guideline

1 Assemble Your BrainLink Pro

1. Connect the Module

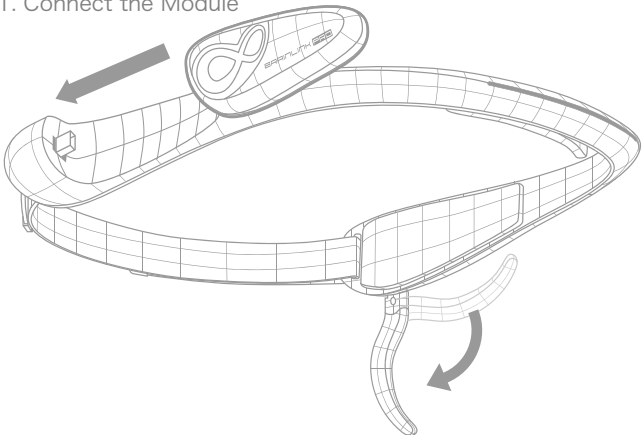


2. Unfold the Back Ear Sensor

2 Assemble Your BrainLink Pro


Top View

1. Insert the Module from a 30 degree angle

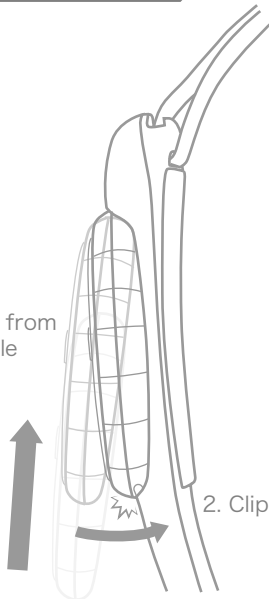


2. Clip the back of the Module to the joint

3 Know Your BrainLink Pro

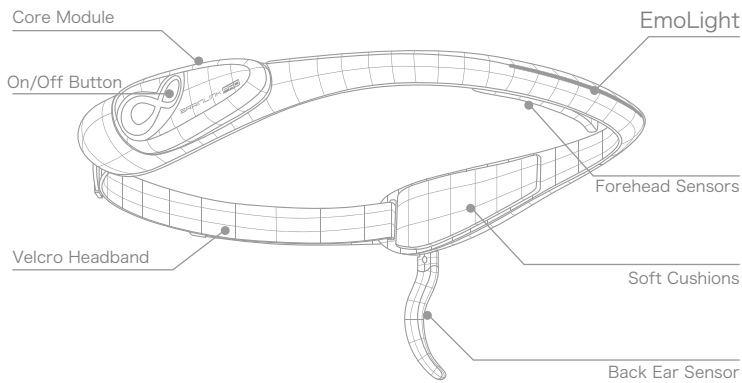

4 Wear


Put on BrainLink Pro from top

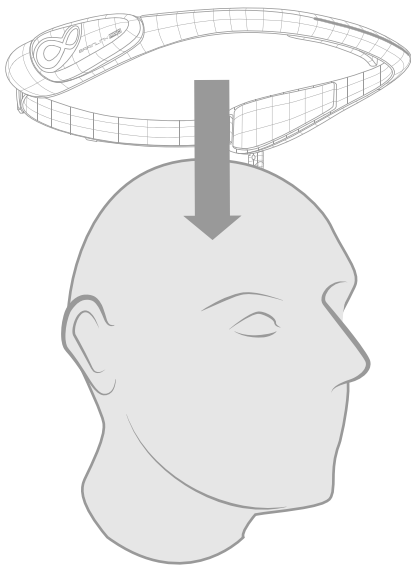

5 Forehead Sensors


Top View

Adjust sensors position and make sure both sensors are touching the forehead firmly

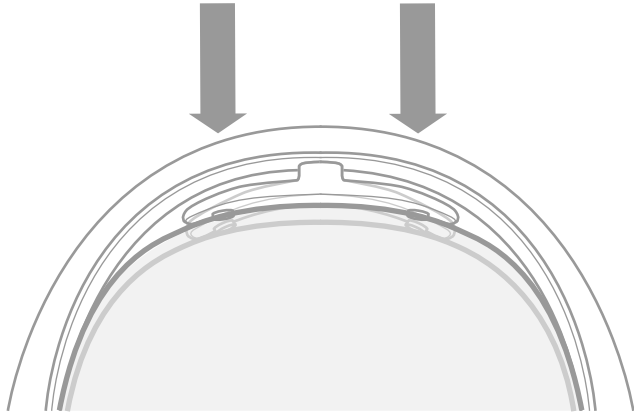

6 Back Ear Sensor & Headband


1. Adjust the Headband tightness & stick it with the velcro

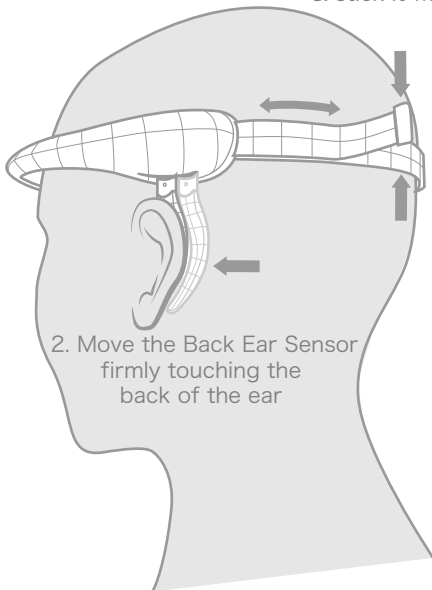

2. Move the Back Ear Sensor firmly touching the back of the ear


7 Signal Check

If you put on BrainLink Pro correctly, you will feel 2 vibrations

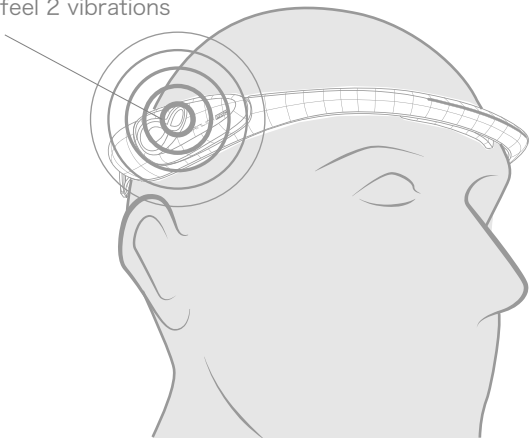


8 Charging

Red light is on when charging
Turns off when battery is fully charged

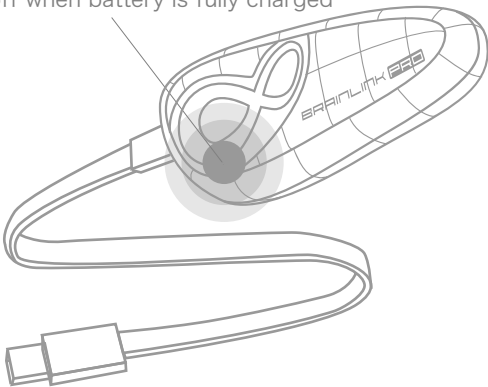

Use 5.0V--1.0A USB power source

9 Bluetooth & Control Tutorial

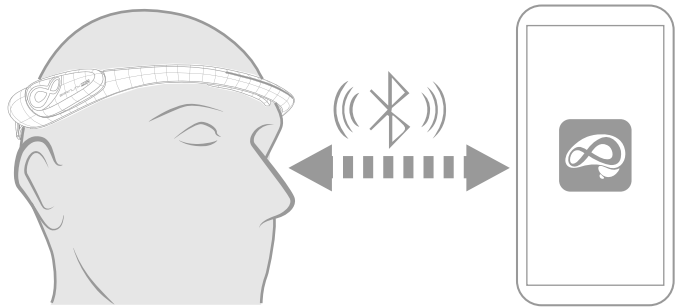

Download the BrainLink Tutorial App for instructions on Bluetooth conection and learn how to use BrainLink to interact in Apps

BrainLink



Apple App Store



Google Play Store

BRAINLINK PRO

Quick Start Guideline