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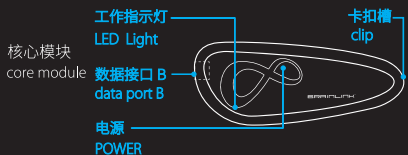
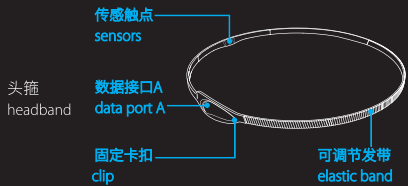
Know Your BrainLink Lite

安装BrainLink Lite
Install your BrainLink Lite

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Plug the data port B into data port A, and make sure the clips are locked.

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Search **BrainLink** on Google Play, SnapPea, Application of treasure and download the Apps.

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If your device doesn't prompt you to download and install the app, go to <http://o.macrotellect.com/app> and you will be redirected to the Apple App Store or Google Play Store depending on your device.

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Connect BrainLink Lite With Your Mobile Device

长按(蜂鸣声后LED红蓝交替闪烁)
long press POWER (after a beep you can see LED Light flashing alternately between red and blue)

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
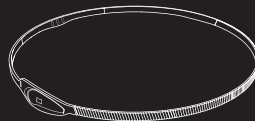
When paired, on iOS device, you will see CONNECTED in MY DEVICES under Bluetooth page. On Android devices however, there will not be any connection information shown on the bluetooth list. The apps will anyway connect automatically.

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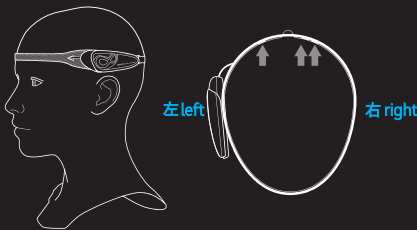
Please make sure that the core module is on the left side of your head, and all 3 sensors are pressing against your forehead firmly.

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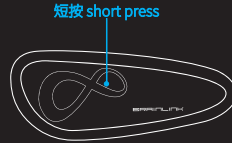When the LED light flashes and beeps continuously, it means the battery of your BrainLink Lite is low.

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You can use the USB charging cable attached in the package to charge your BrainLink Lite to any USB port. The red LED light will be off when the charging process completes.
Note: Please keep the USB charging cable safely. It is a customized cable by Macrotellect and BrainLink Lite is not compatible to any other micro USB or mini USB cable.

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Product Attributes: Brainwave sensing Headset

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Battery: Rechargeable Lithium battery

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Blue Light On: Bluetooth successfully connected

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Manufacturer:Shenzhen Macrotellect Ltd.