



EXCELLENT BRAIN

Pairing and Activation Guide for: EEG sensor - Neurosky MindWave Mobile Windows 10 vresion





SETP - 1

- Insert AAA battery to the appropriate location in the headset.
- Turn the sensor “on” and make sure a blue light appears.
 - *Red light indicates that the battery should be replaced.






STEP - 2

Sensor - Bluetooth pairing


- Select the START on your PC menu and then open the Settings menu, as shown below:

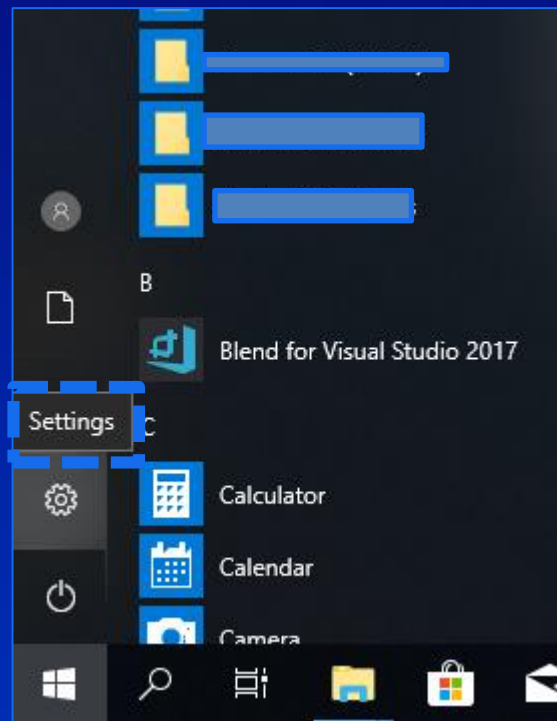



STEP - 3


- Select the “Devices” option

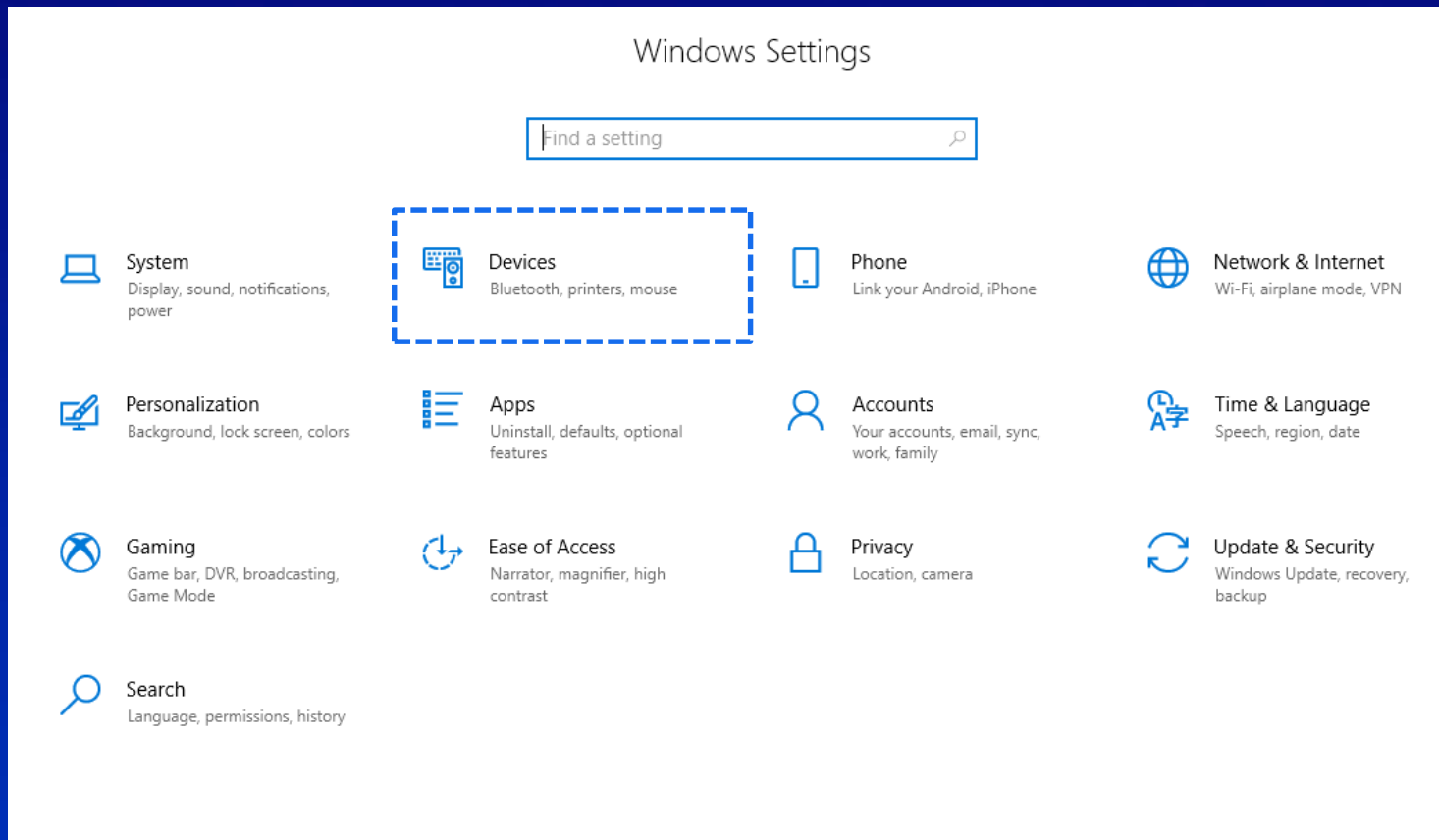



STEP - 4


- At the top of the window that will pop-up select the (+) sign “Add Bluetooth or other devices”

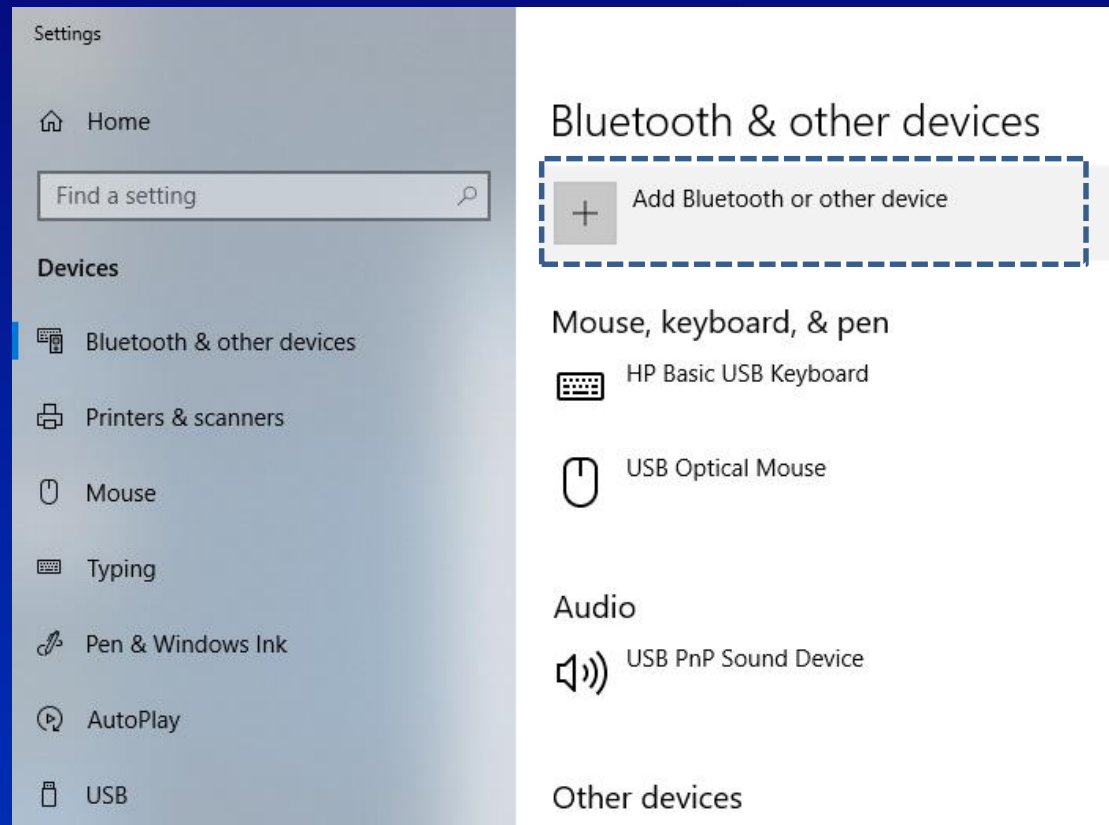



STEP - 5



- At the “Add device” menu that will pop-up select the first option: “Bluetooth”.

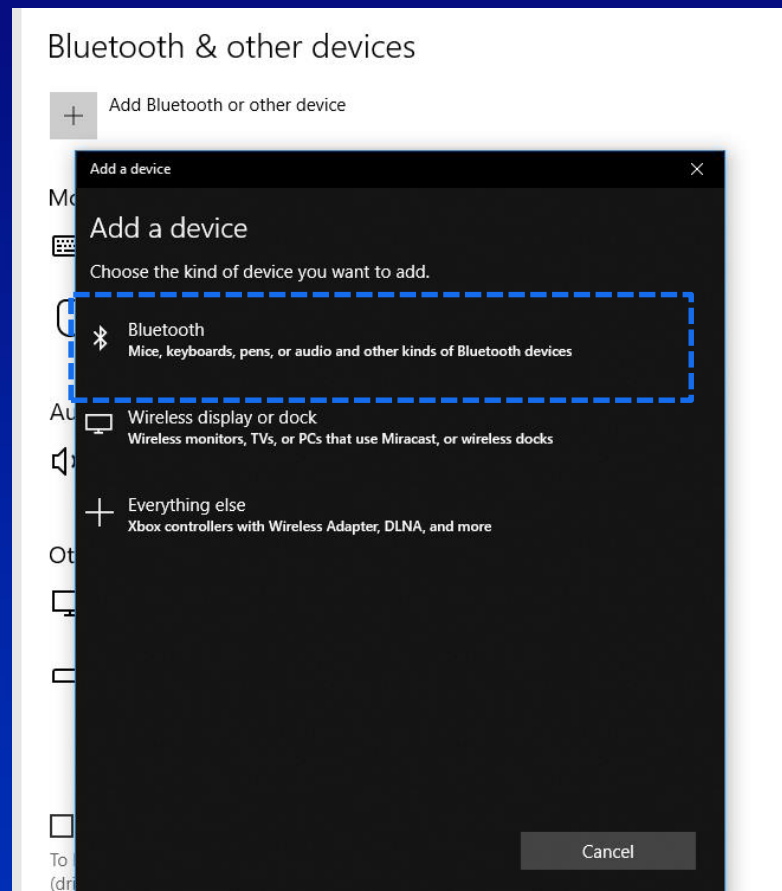



STEP - 6

- In the next list, select “*MindWave Mobile*” with the headphone icon
- *At the first connection some users will see “unknown device” instead with the same headphone icon.*

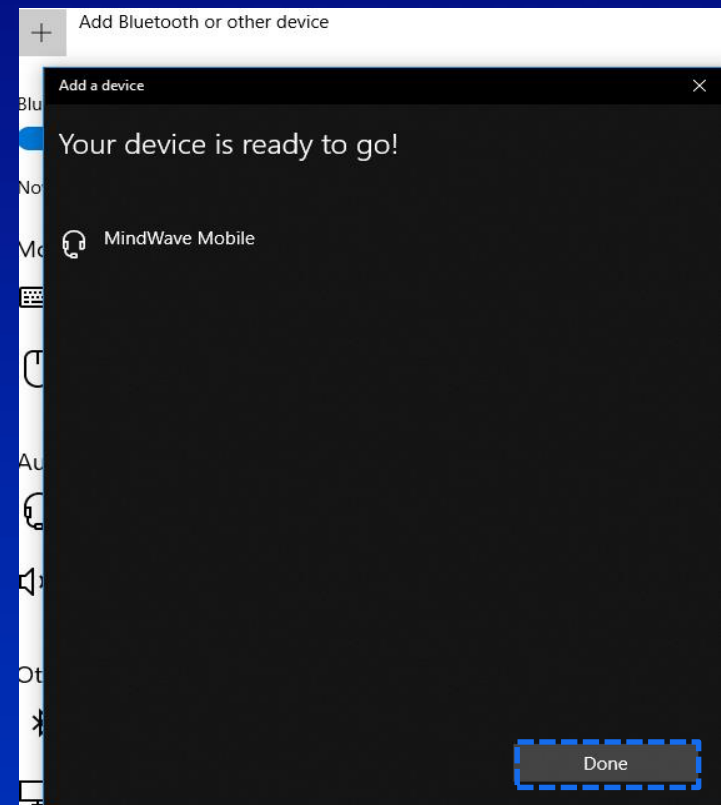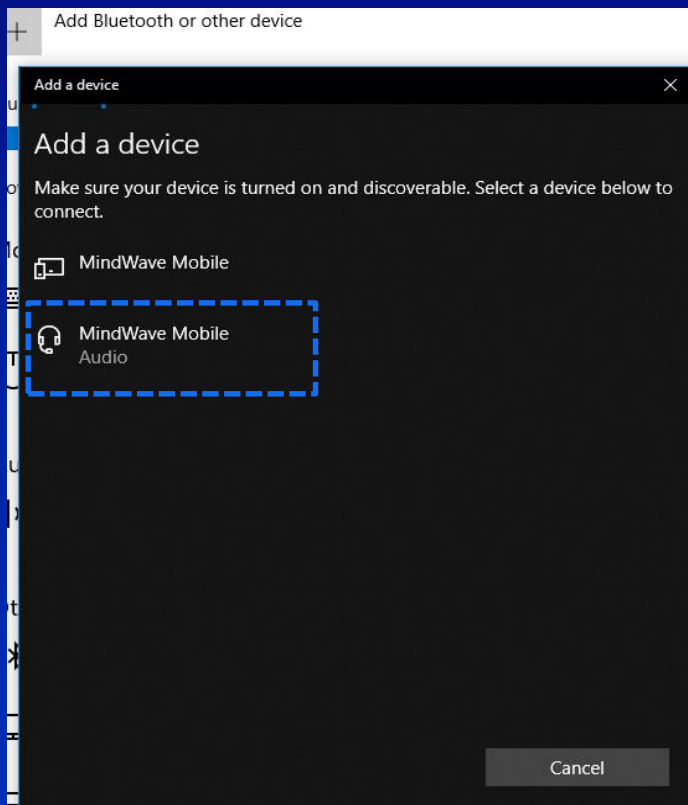



STEP - 7

- The pairing process will start with a: “Connected” status and will be changed shortly to paired.
- Now the connection is completed




MindWave Mobile
Paired




The EEG sensor

Headset Adjustment

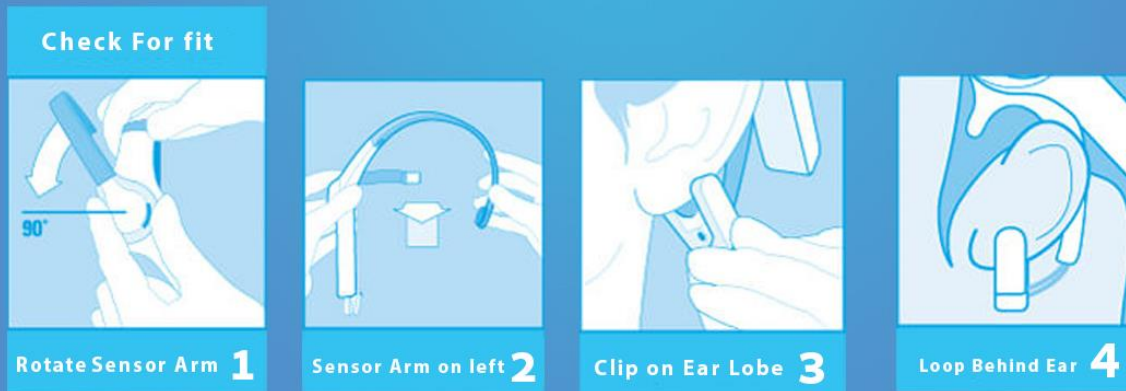



Headset Placement

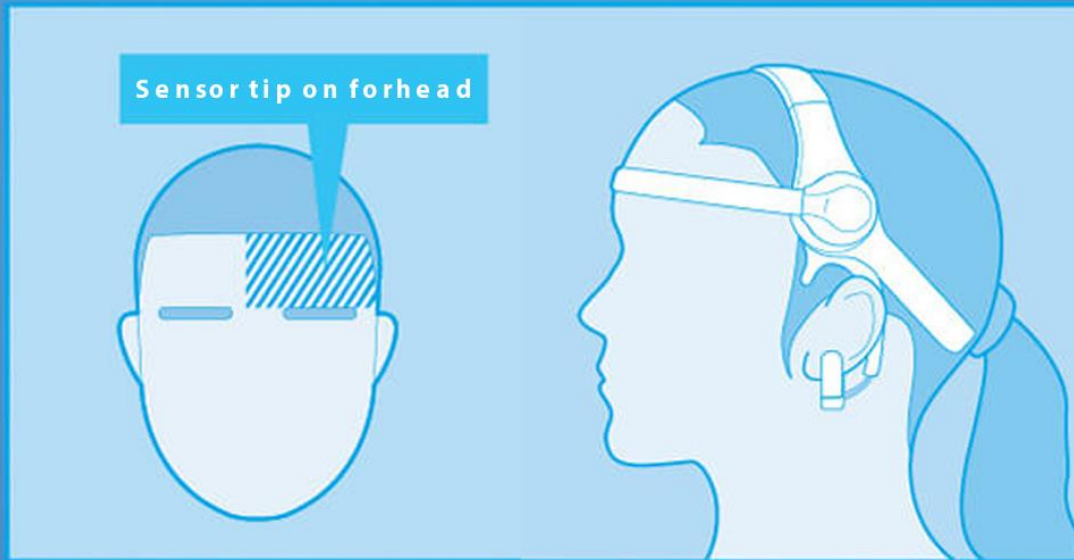



Improving Reception Quality



Squeeze Ear clip
to press contacts
to touch ear



Press sensor to
forehead to
skin contact



For better connection

- Be sure to pair the Bluetooth before starting the Excellent-Brain™ program.
- periodically clean the sensor , especially the metallic part that touches the forehead.
- Makeup can affect signal quality from the sensor.
- It is recommended to make sure that the forehead area where the sensor touches is clean and dry.
- Be sure to remove earrings from the left ear.
- Cellphones and devices with high radiation emissions may have an effect on the results and should be kept away from the sensor.



EEG Reception Quality

After starting the Excellent-Brain™ software, you can check the quality of the signal coming from the sensor at the bottom of the screen.

- **Red**: no signal, see “[Sensor - Bluetooth pairing](#)” or replace the AAA battery.
- **Yellow** - sensor not fully aligned. See “[Headset Placement](#)”/“[Improving Reception Quality](#)”.
- **Green** – everything is connected, you can start training your Excellent Brain™.





EXCELLENT BRAIN

The Sensor In Action





EXCELLENT BRAIN

Our technical support team is here for you:

***Phone* : (+972)77-2042282**

***by mail* : support@excellent-brain.com**

***on our website* : www.excellent-brain.com**