
Part description

Accessories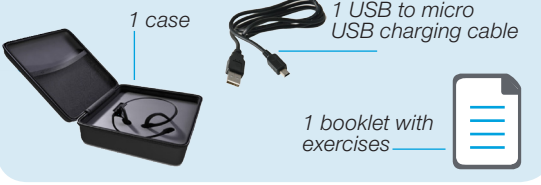

Features

- Bone conduction headphone
Very sensitive microphone
Dynamic filter to create a perceptive contrast

How to wear it

Place the wraparound head band behind your neck with the transducers resting on the bone in front of your ears...

How to charge

Connect FORBRAIN to a standard USB port on your PC or a wall charger via the USB charging cable.

LED Indication

Table with 2 columns: Color (Solid Blue, Blue light goes out, Solid Red, Red light goes out) and Status (Normal working mode, Battery too low, Headphone is being charged, Charging completed)

Troubleshooting

Troubleshooting table with columns: Issue (Does not turn on, Don't hear my voice, Have some feedback, Sound is too loud), Possible Problem, Resolution

Specifications

Specifications table: Speaker Type, Sensitivity, Microphone type, Sensitivity, Playback time, Charging time, Weight, Color, Carton size


Storage and maintenance

- Store the headset in cool and dry place. The working temperature should be 0-45°C...
After storage for a long period, please charge the headset before using again.

FCC Warning

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference...

Where is the serial code?

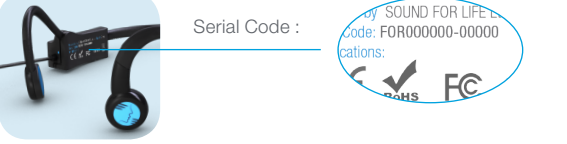Precaution of use, limitations and disclaimer

You do not need to change the volume of FORBRAIN with few rare exceptions (very poor or hypersensitive hearing). Do not turn the volume of FORBRAIN up too high.

How to contact us?

SOUND FOR LIFE Ltd
Unit 1010, 10/F, Miramar Tower
132 Nathan Road
Tsim Sha Tsui, Kowloon, Hong Kong

Note: If you need to return a product under warranty, please send a detailed explanation of the problem to: info@forbrain.com

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