EN Warnings

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Do not incinerate the device.

Firstkind Limited*

Hawk House
Peregrine Business Park
High Wycombe
HP13 7DL
United Kingdom

T: +44 (0)1494 572040 www.gekodesigns.com

If assistance is required or to report difficulties, unexpected operation or events, contact the manufacturer.

is a choke and potential allergic hazard. If the device or any component is swallowed seek IMMEDIATE medical assistance. Serious harm could be caused if the battery is swallowed.

Check the device is functioning correctly every 2 hours.

Use the device only as instructed. Visit www.gekodesigns.com for further advice and usage tips. Keep out of the reach of children.

Read the instructions carefully before use. Only apply the device in accordance with instructions. Improper use of this device can be dangerous.

The geko™ device may be operated either by the patient or a third party. Patients should be trained in the positioning and operation of the device.

Keep out of the reach of children and pets.

No modification of this equipment is allowed.

Do not use whilst driving, operating machinery, or during any activity in which involuntary muscle contractions put the user or others at undue risk of injury.

Excessive force may damage the device.

EMC Declaration

The geko™ device is certified to EN 60601-1-2:2007 regarding Electromagnetic Compatibility. Medical Electrical Equipment needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided at: www.gekodesigns.com

Disposal
The device does not contain any toxic or environmentally hazardous materials. After use the device may be potentially contaminated or infected because it has been in contact with skin for several hours, and so need to be disposed of with care.

Batteries must be disposed of in accordance with any local legislation. Some hospital and clinics will have specific requirements for disposal of used medical devices. If used at home the device maybe disposed of in your general waste if regulations permit. For ease of disposal the electrodes may be removed with scissors if necessary. The battery can be removed by breaking open the housing and prising out. See www.gekodesigns.com for guidance.

Do not apply over or, in proximity to, cancerous lesions.

Do not use in proximity (i.e. within 1m) of short wave/ microwave equipment, as this may affect the device.

Portable and mobile RF communications equipment can affect medical electrical equipment.

Do not use if packaging is open or appears damaged.

Do not use a device that appears to be damaged.

Do not use the device in close proximity to heat sources, such as fires or radiant heaters, as excess heat may affect the performance of the device.

In some cases skin inflammation or irritation can develop in the contact area:

The hydrogel contains acrylic polymers. Do not use on patients known to be allergic to acrylic acid.

or following surgery where muscle contractions may disrupt the healing process.

The device should be removed before the patient undergoes MRI as it contains ferromagnetic components.

The device should be switched off during ECG monitoring using leg electrodes as it may interfere with ECG leg electrode signals.

The device is not intended for use in an oxygen rich environment.

Do not place the device in the mouth. The device

is a choke and potential allergic hazard. If the device or any component is swallowed seek IMMEDIATE medical assistance. Serious harm could be caused if the battery is swallowed.

Check the device is functioning correctly every 2 hours.

Use the device only as instructed. Visit www.gekodesigns.com for further advice and usage tips. Keep out of the reach of children.

Read the instructions carefully before use. Only apply the device in accordance with instructions. Improper use of this device can be dangerous.

The geko™ device may be operated either by the patient or a third party. Patients should be trained in the positioning and operation of the device.

Keep out of the reach of children and pets.

No modification of this equipment is allowed.

Do not use whilst driving, operating machinery, or during any activity in which involuntary muscle contractions put the user or others at undue risk of injury.

Excessive force may damage the device.

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