


GETTING STARTED


How to record your first ECG

1. MEET YOUR KARDIA MOBILE ECG

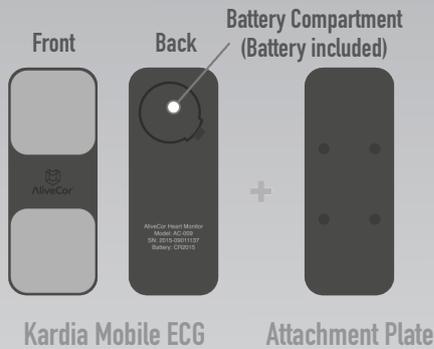


Unpack

Take your Kardia Mobile ECG out of the box. The Mobile ECG is already connected to an attachment plate that can adhere to the back of your smartphone or tablet or preferred device case.

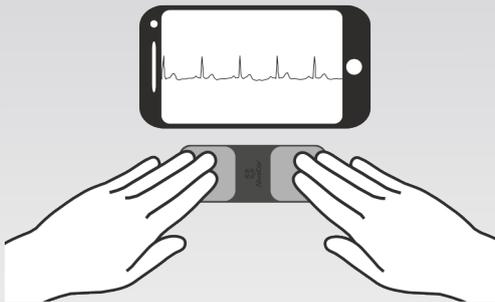

TIP: Some customers prefer not to attach the Kardia Mobile ECG to their smartphone or tablet (or device case), which is just fine with us! Your Mobile ECG will work normally when it is placed within 12 inches (30 cm) of your smartphone or tablet.

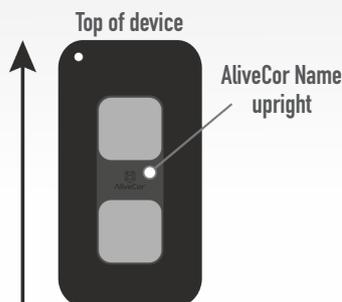


Attach (Optional)

Direction matters! Peel off the white strip from the back of the attachment plate, and stick the plate securely to the center back of your smartphone or tablet (or device case), checking to make sure that the word "AliveCor" in the center of the Mobile ECG is upright. In doing this, the electrode (silver plate) above the word "AliveCor" will be placed near the top of your device and the electrode below the word "AliveCor" will be placed near the bottom of your device.



TIP: If you buy a new smartphone or tablet or device case, your Kardia Mobile ECG can go with you! To detach your Mobile ECG from the attachment plate, use your thumbs to slide the Mobile ECG toward the open end of the plate. Though the adhesive is designed for one-time use, additional attachment plates are available for a low price at store.alivecor.com in the U.S. or through one of our international distributors.

2. DOWNLOAD THE ALIVEECG APP

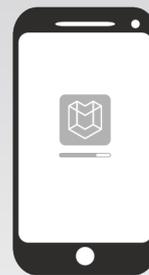

Create an account


First, ensure your smartphone or tablet is connected to the Internet. Search for AliveECG in the Google Play™ store or App Store. Download and install the AliveECG app. If using an iPad, search for the AliveECG app for iPhone in the App Store.

TIP: If prompted during setup to enable the AliveECG app to access the microphone, please allow this access. Your Kardia Mobile ECG works with the microphone on your smartphone or tablet. Open the AliveECG app and follow the instructions to set up an AliveCor account. As a part of the account set-up, you will be asked to take your first recording.

3. RECORD YOUR FIRST ECG

Right Hand

Left Hand

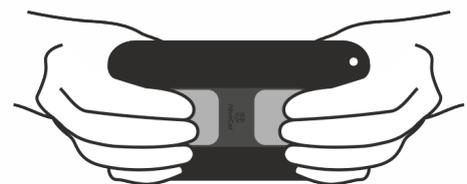

Position your fingers

Place two or more fingers (it doesn't matter which fingers) from each hand on the electrodes for 30 seconds. While recording, try to relax your arms and hold still.

TIP: Each time you record, your left hand should touch the electrode above the word "AliveCor" and your right hand should touch the electrode below the word "AliveCor."

The AliveECG app makes it easy to share ECGs with anyone by emailing your recordings. A more efficient alternative for medical professionals to receive ECG recordings from their patients using the Provider Dashboard. The Provider Dashboard is a free web-based application to help you review your patients' ECG data. Once you connect to a patient in the Provider Dashboard, their future recordings automatically start showing up. Then, you can quickly review the metrics that you deem most important (e.g. symptoms, ECG analysis reports, heart rate).

1. PATIENT RECORDINGS

Each patient you connect with will have their recordings listed under their profile page. Reviewing ECGs enables you to:

- Easily identify the recordings you have not yet reviewed
- Flag noteworthy recordings
- View reports for a particular recording
- View the symptoms your patients were feeling during the recording

FLAG	DATE & TIME	SYMPTOMS	BPM	RECORDING	ECG ANALYSIS REPORTS
🔖	Apr 25th, 2014 10:46AM	chest pain, shortness of breath	81	VIEW ECG	
🔖	Apr 25th, 2014 10:46AM		58	VIEW ECG	
🔖	Apr 25th, 2014 10:46AM	chest pain, palpitations, shortness of breath	118	VIEW ECG	
🔖	Apr 25th, 2014 10:46AM	chest pain, shortness of breath	104	VIEW ECG	

2. ECG ANALYSIS

The Provider Dashboard enables you to quickly see which of your patients are sending ECGs for analysis. On the „Recent Reports“ tab, you can see any reports that have come back in the past 2 weeks.

FLAGGED	LAST NAME	FIRST NAME	DATE	FINDING	SEVERITY	BPM	SYMPTOMS
🔖	Smith	John	Apr 22nd, 2014 02:59PM	2nd Degree AVB Type I - Sinus Bradycardia w/ 1st Degree AVB/3.5 Second Pause	●	76	chest pain, dizziness, fatigue, palpitations, shortness of breath

You can view reports for a particular patient on the patient page.

FLAG	DATE & TIME	FINDING	SEVERITY	BPM	SYMPTOMS
🔖	Mar 24th, 2014 01:58PM	Sinus Rhythm-Sinus Bradycardia w/ Bigeminal PVCs/PVCs/Lead Loss		62	fatigue

3. NOTIFICATIONS

As you set up a patient on your dashboard, you have the option to be notified (via email) when they receive their ECG analysis report

Patients Recent Reports Invite A Patient Archived Patients

← back to all patients Disconnect

Kariane Hilll
35 years old

Email me when this patient receives an ECG analysis report

Note: Information is provided by the patient from the mobile app.

4. THE FIRST STEPS

In order to view a patient's recordings, you must send them an invite via email. To send an email invite:

1. Go to www.alivecor.com
2. Click on „Sign in“ in the upper right corner and enter your email address and password
3. Click on „Invite a Patient“
4. Type in the patient's email address (this must be the email address they use for their AliveCor account)
5. Click „Send Email“

5. CONCLUSION

The Provider Dashboard is designed to make the process of reviewing patient data as efficient as possible. Try it out today and let us know what you think!

If you have any problems with the AliveCor Mobile ECG please contact us under:



TITAN Commerce Continental Services GmbH
 Gottlieb-Daimler-Straße 13
 35440 Linden, Deutschland
 Telephone: + 49 (0) 6403-60993-0
 Fax: + 49 (0) 6403-60993-01
 info@titan-commerce.com
 www.titan-commerce.com
www.mindtecstore.com/en/shop/alivecor