



Powering the Neeuro SenzeBand On/Off

To switch the SenzeBand on, press the power button once. Wait for the green light to blink two times. A static green light indicates that the SenzeBand is switched on.

To switch the SenzeBand off, press the button once while the green light is on. The green light will turn red and blink two times before switching off.

Charging the Neeuro SenzeBand

Switch the SenzeBand off, plug the Micro-USB charging cable that is included in the package into the SenzeBand charging port. A red light indicates that the SenzeBand is charging, a green light indicates that the SenzeBand is full charged.

Setting Up the SenzeBand

Step 1: Download a Neeuro App from the Apple App Store or Google Play Store.

Step 2: Ensure the Bluetooth function is enabled on your phone or tablet device. For iOS and Android devices, go to Settings > Bluetooth.

Step 3: Switch on the SenzeBand, a static green light indicates the SenzeBand is switched on. A red light indicates the SenzeBand is out of battery and needs to be charged.


Step 4: Open the Neeuro application and login to your account. In the application, go to Settings > Device > Add Device and follow the instructions to connect to your SenzeBand.

Step 5: Ensure that your forehead and sensors on the SenzeBand are wiped clean. The user of non-alcoholic wipes are recommended.

Step 6: Place the SenzeBand on your forehead, ensuring the sensors sit around an inch above your eyebrows. Placing the sensors too high above the forehead may result in inaccurate readings or unstable connection.


2

Step 7: Adjust the back reference sensors such that they are hooked securely behind both ears.

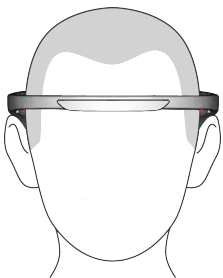


Frontal View

Ensure the SenzeBand strip sits about one inch above the eye brows.

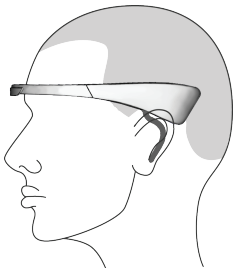

Side View

Ensure the two reference sensors touch the back part of the ear completely and securely.

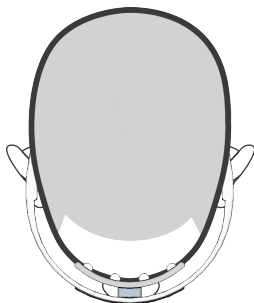

Top View

Ensure the four front sensors touch the skin of the forehead firmly.

3

Step 8: To ensure that you are properly connected, under the Neeuro App, go to Settings > Status, adjust the SenzeBand until all four indicators turn green, indicating the sensors on the SenzeBand have good connection in reading your brainwaves.

Enjoy using your Neeuro SenzeBand.

4

Safety and Compliance

Do not use your SenzeBand device in wet conditions or rain. If the SenzeBand gets wet, unplug all cables and turn off the device. Wait for the SenzeBand to dry completely before pressing the power button to turn the SenzeBand on again. Do not attempt to dry the SenzeBand through external heat sources such as hair dryers. Do not expose the SenzeBand to extreme heat or cold. If further support is needed, please contact us at support@neuro.com.

Battery Safety

The rechargeable battery in the SenzeBand should not be repaired or replaced by any third-party. The SenzeBand should only be charged using a Micro-USB cable such as the one provided in the package.

FCC Caution

Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operations.

FCC Radiation Exposure Statement

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment.

This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

WEEE Compliance

The SenzeBand must not be disposed of with normal household waste. The SenzeBand must be disposed of by arranging to return to designated collection points for the recycling of electrical waste and electronic equipment. Separating and recycling the SenzeBand at the time of disposal will help to conserve natural resources and ensure that the equipment is recycled in a manner that protects human health and the environment.

Neeuro Pte Ltd (Reg: 201302274H)

79 Ayer Rajah Crescent, #02-19
LaunchPad @ One-North
Singapore 139955

support@neuro.com