

SenzeBand Activation


User Guide to activate your SenzeBand and upgrade
your Basic account to Premium account

for Memorie version 1.1
updated 25 November 2016






Search for “Neuro Memorie” on Apple App Store or Google Play Store
Install **Memorie** app

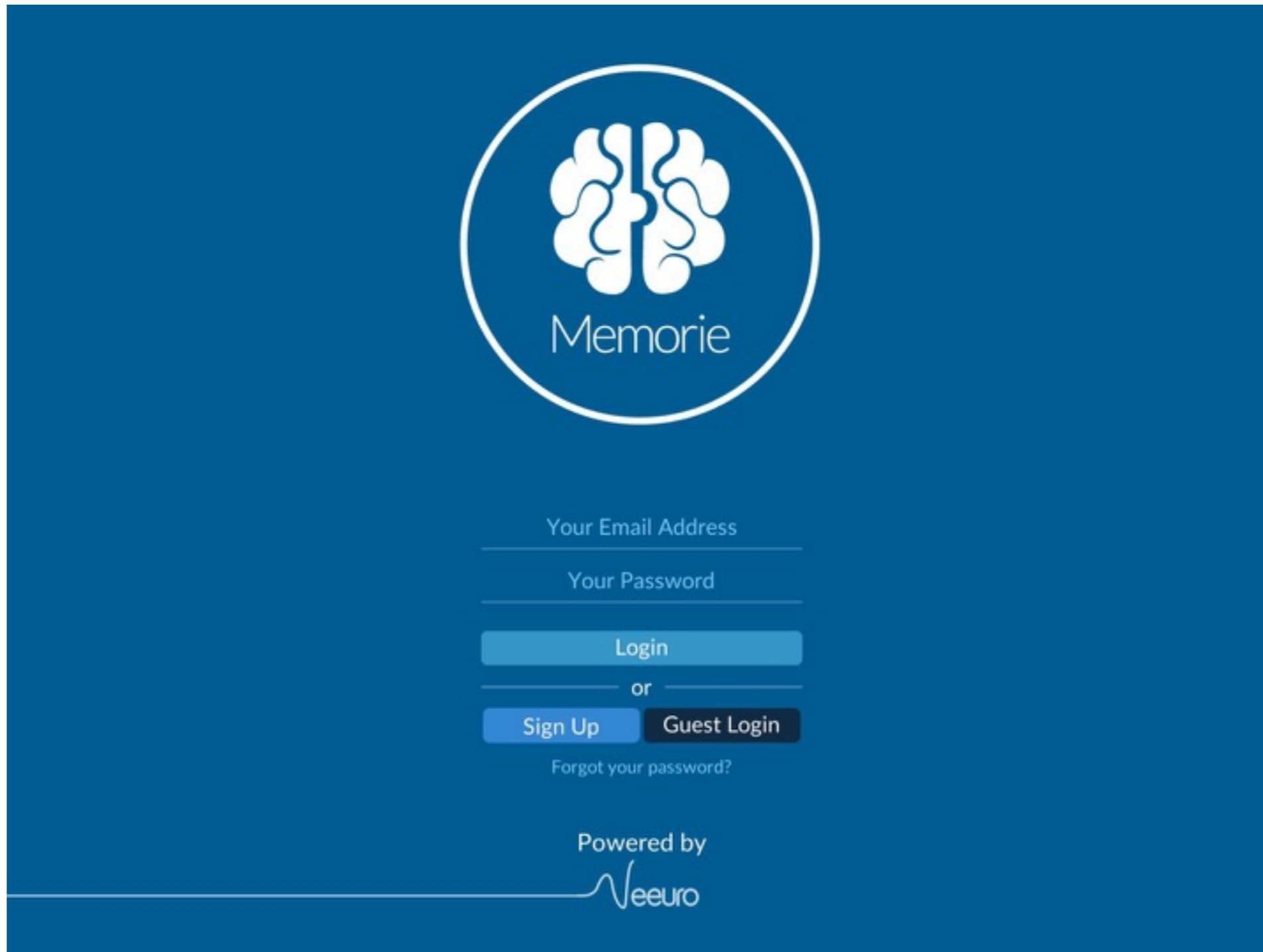


Start **Memorie** app


Ensure you have Internet connection + Bluetooth is enabled

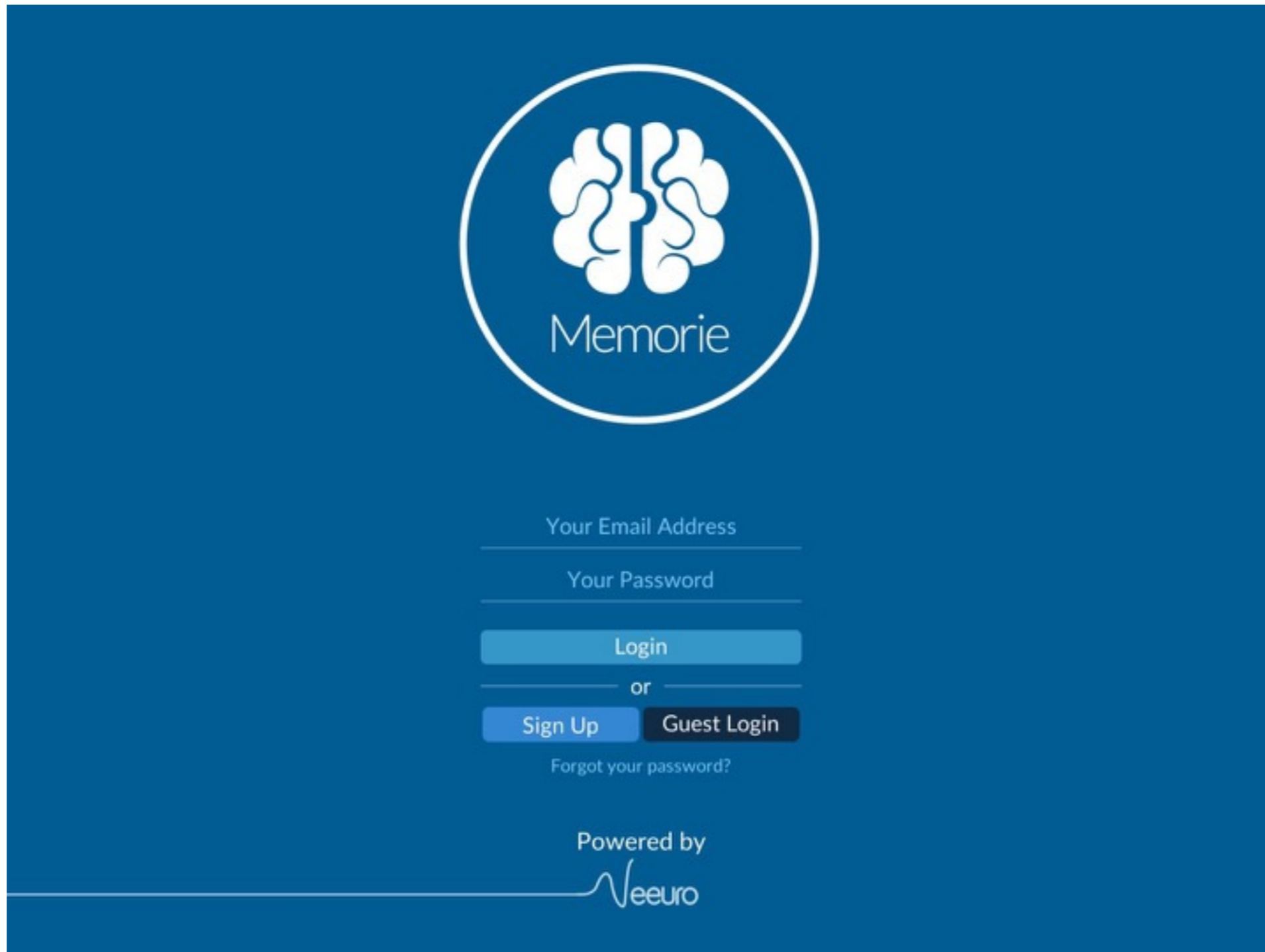



Tap on **Sign Up**


Sign Up



Name * 


Email * 

Create Password * 

Retype Password * 

Gender 

Birth Date  


Country 

* required field

Please read carefully and accept the terms of use:

Email me about new products, features and


Enter your details to sign up
Confirm account from email link

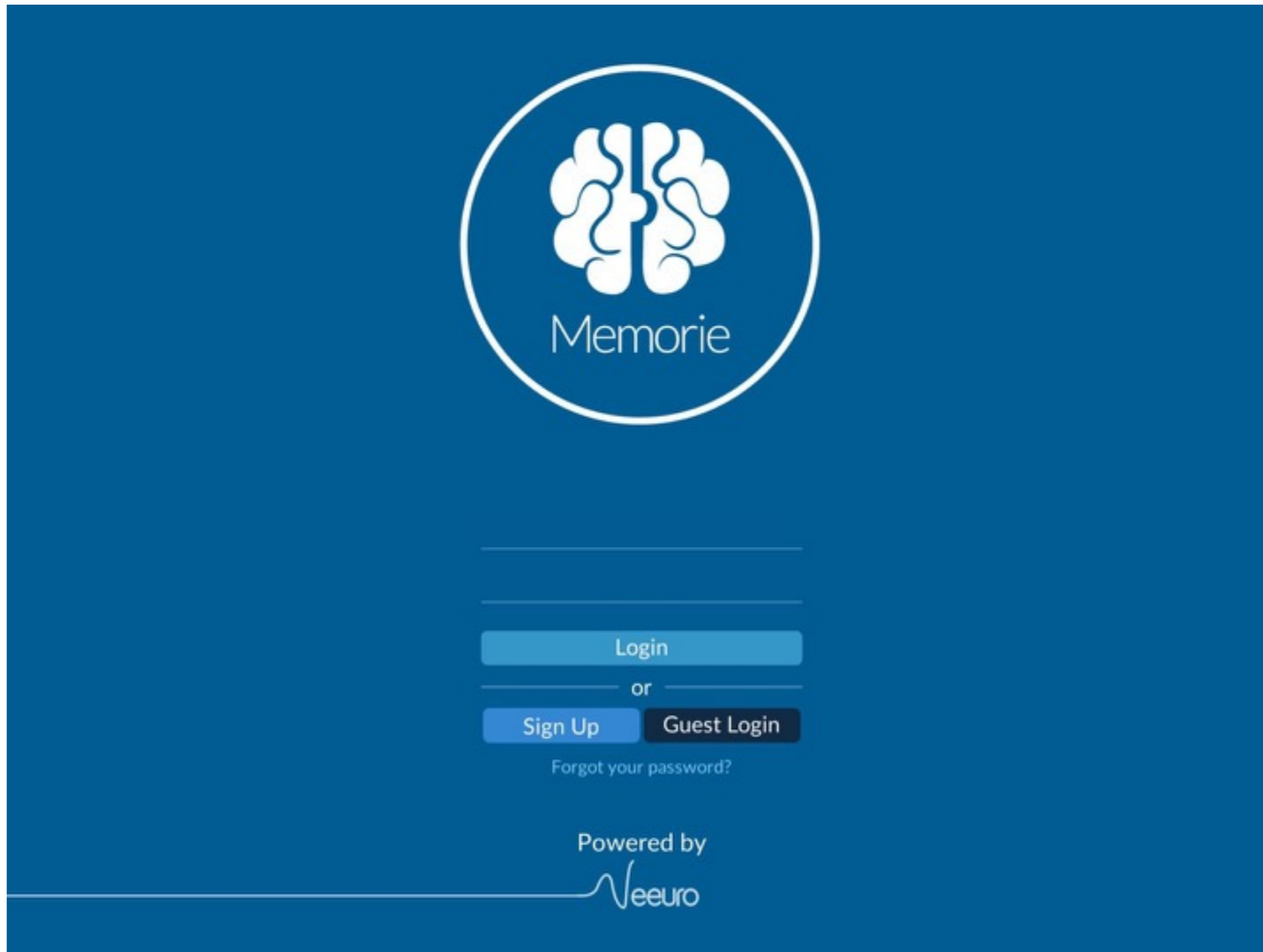


Login with your registered account



Loading data, please wait...

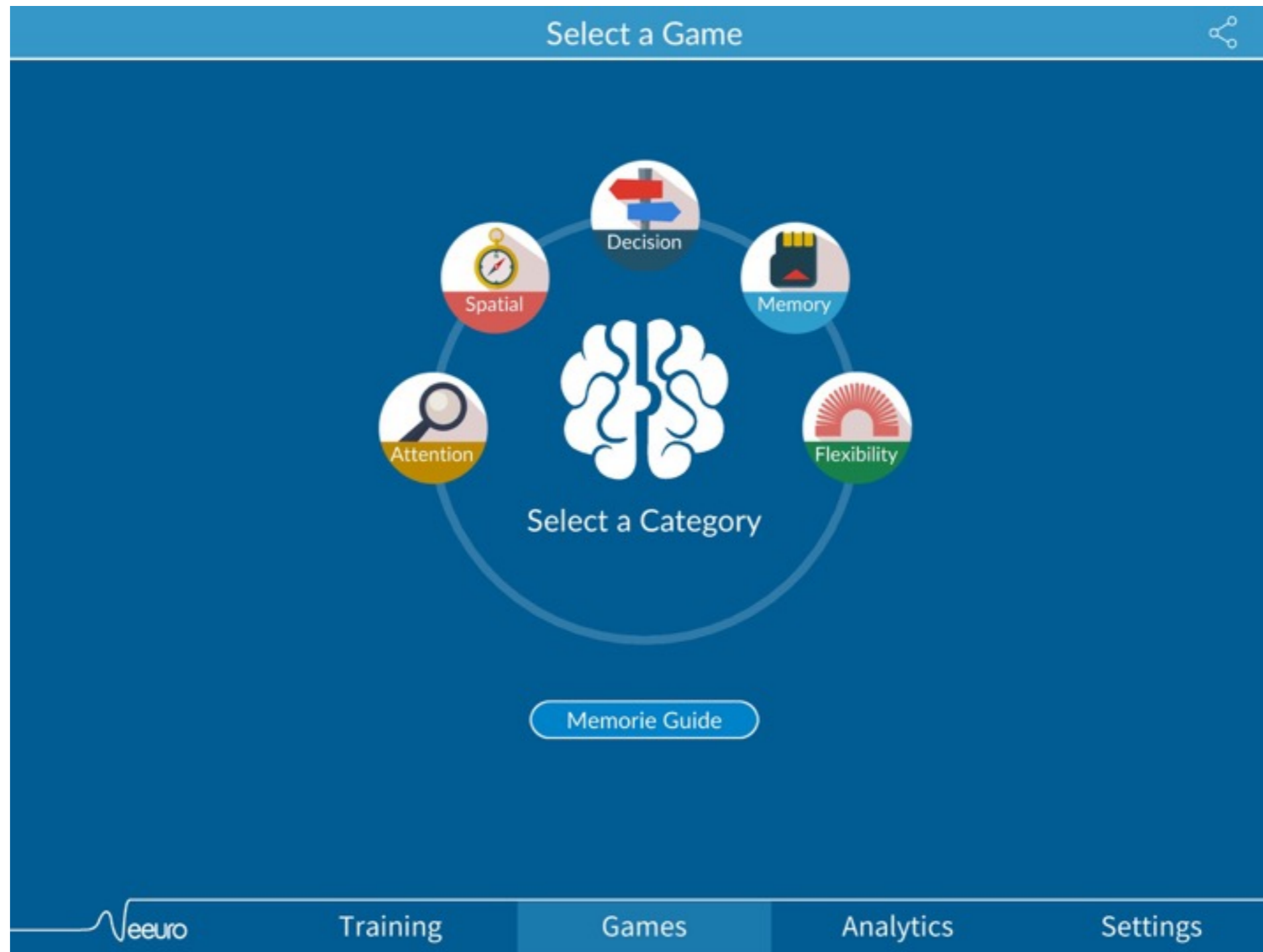

Ready to continue
Tap on **Settings**

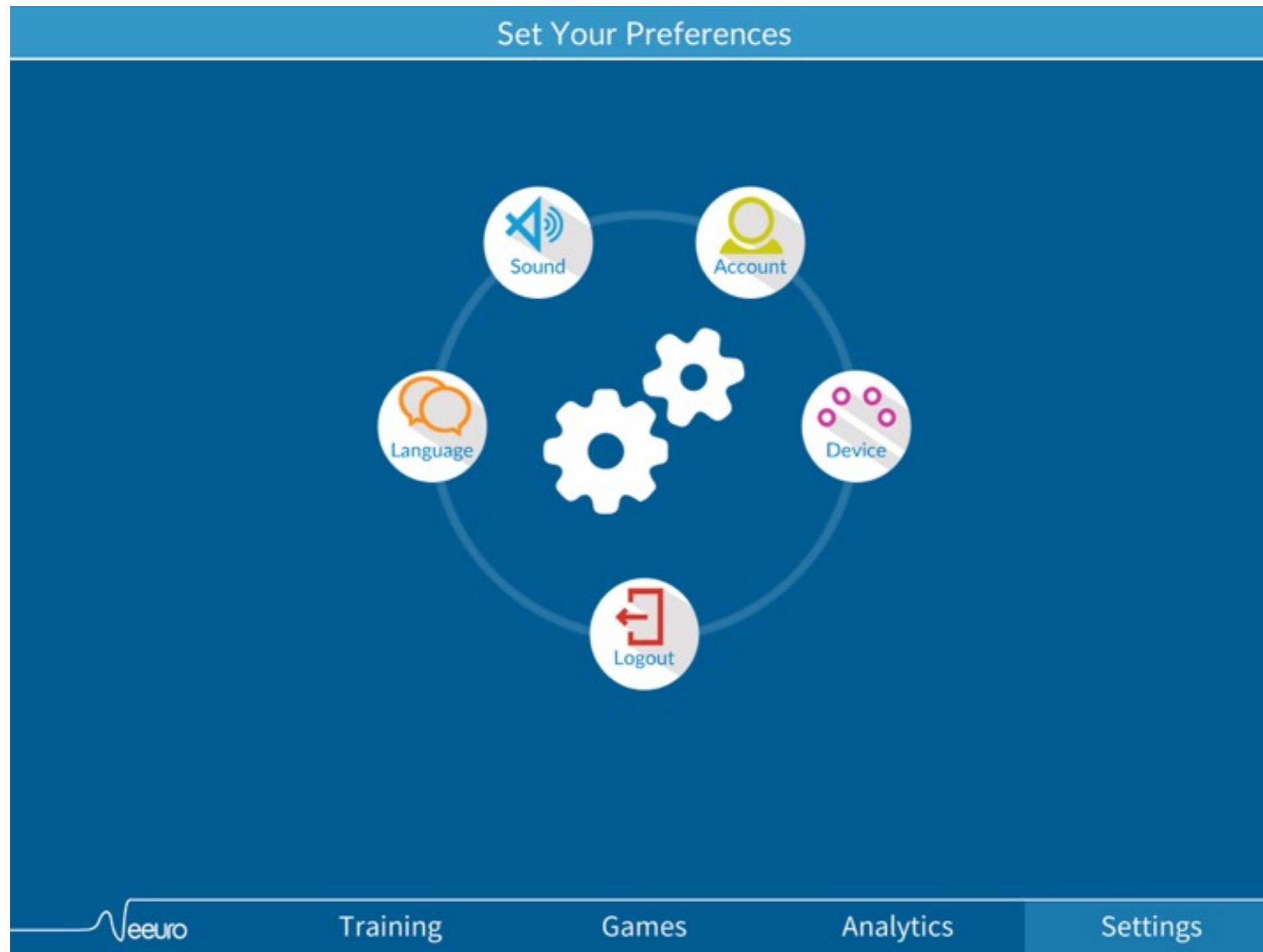


Tap on **Account**


Set Your Preferences


Detail


Account Basic



Email *

Name * 

Change Password * 

Retype Password * 

Gender 


Birth Date  

Neuro Training Games Analytics Settings


Ensure your Account is **Basic**
Tap on Detail to return to Settings




Tap on **Device**

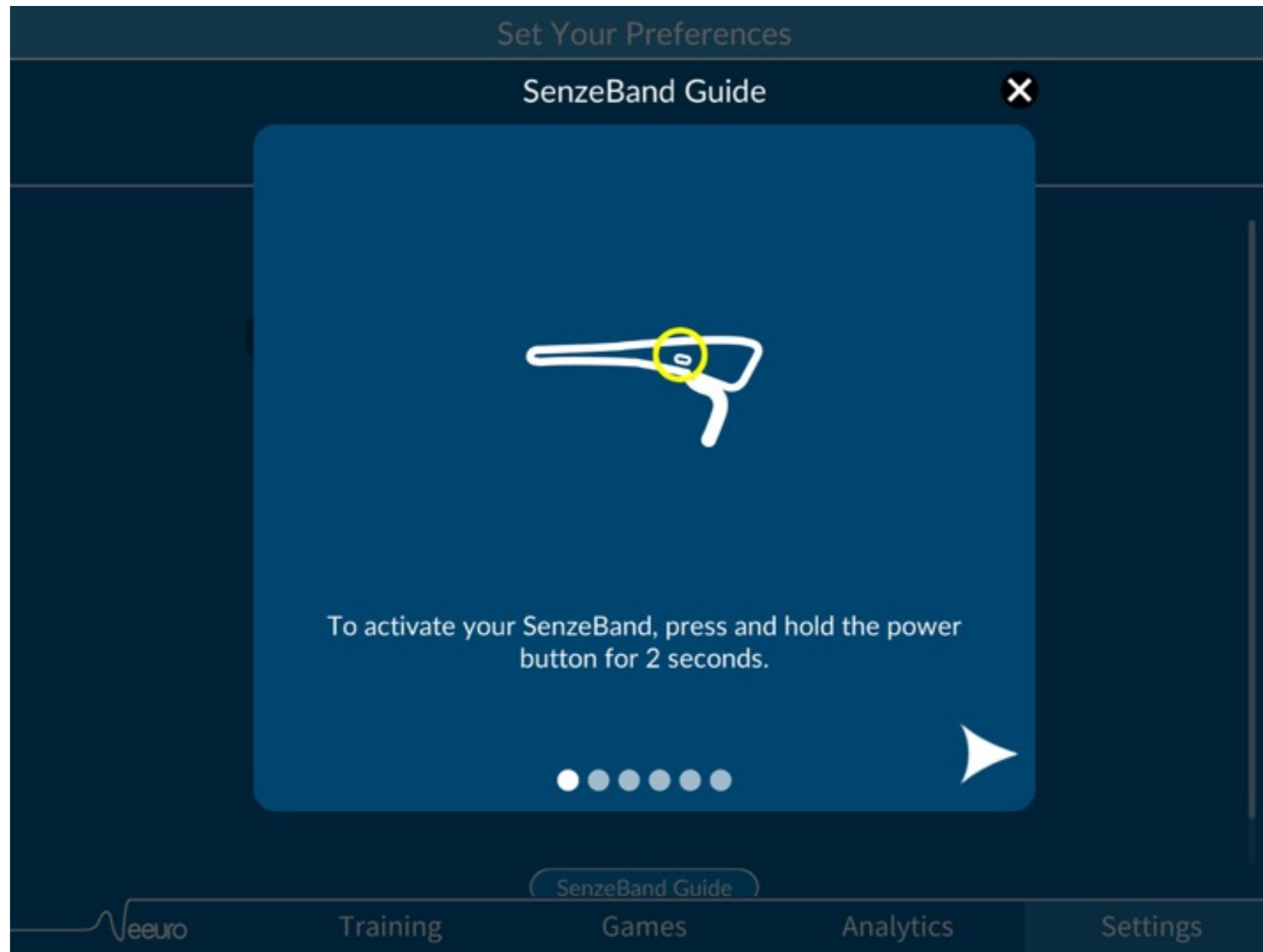

SenzeBand Guide comes up for 1st time user

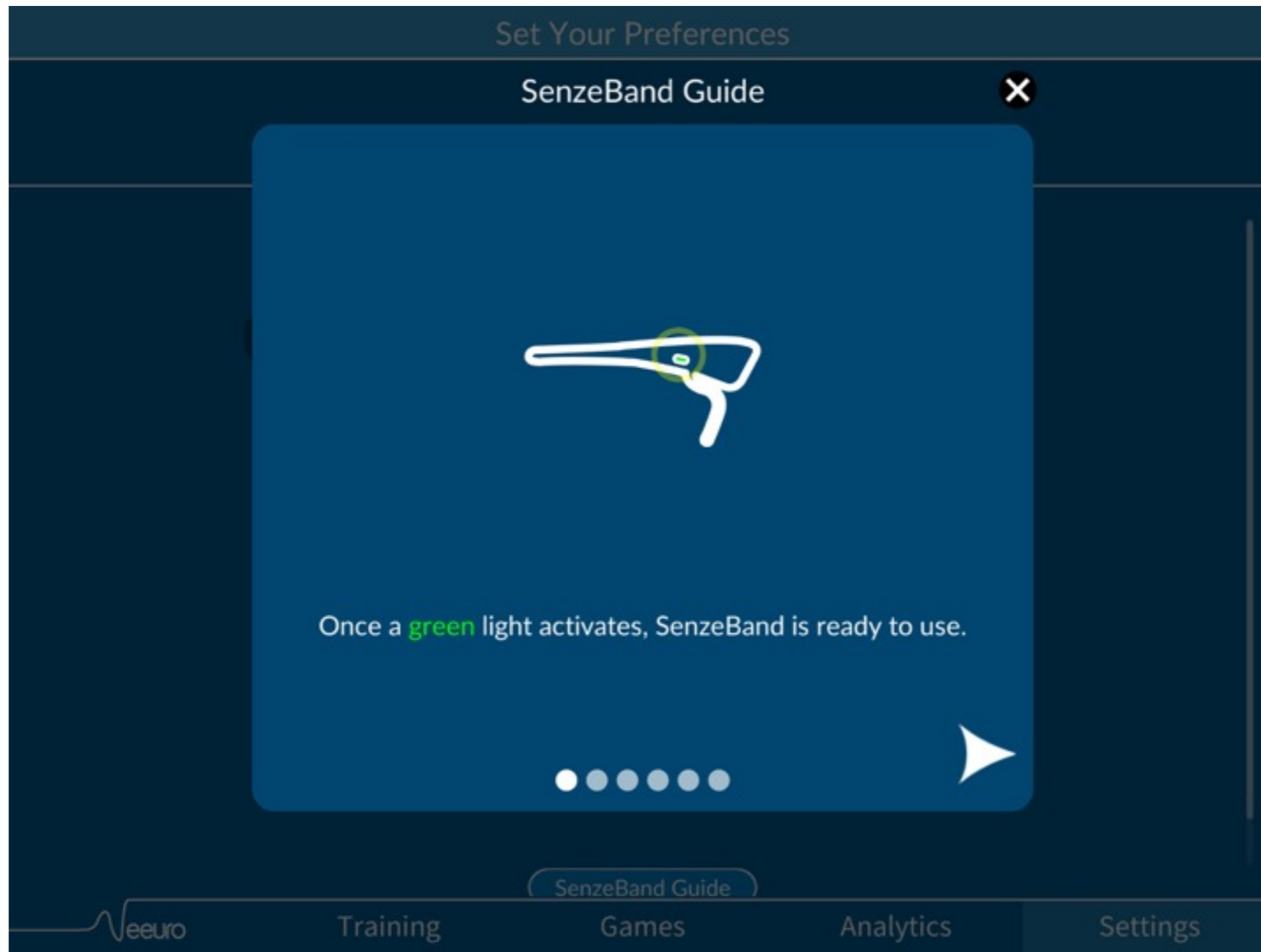

SenzeBand Guide shows how to use the device
Tap on **X** to close the Guide after viewing

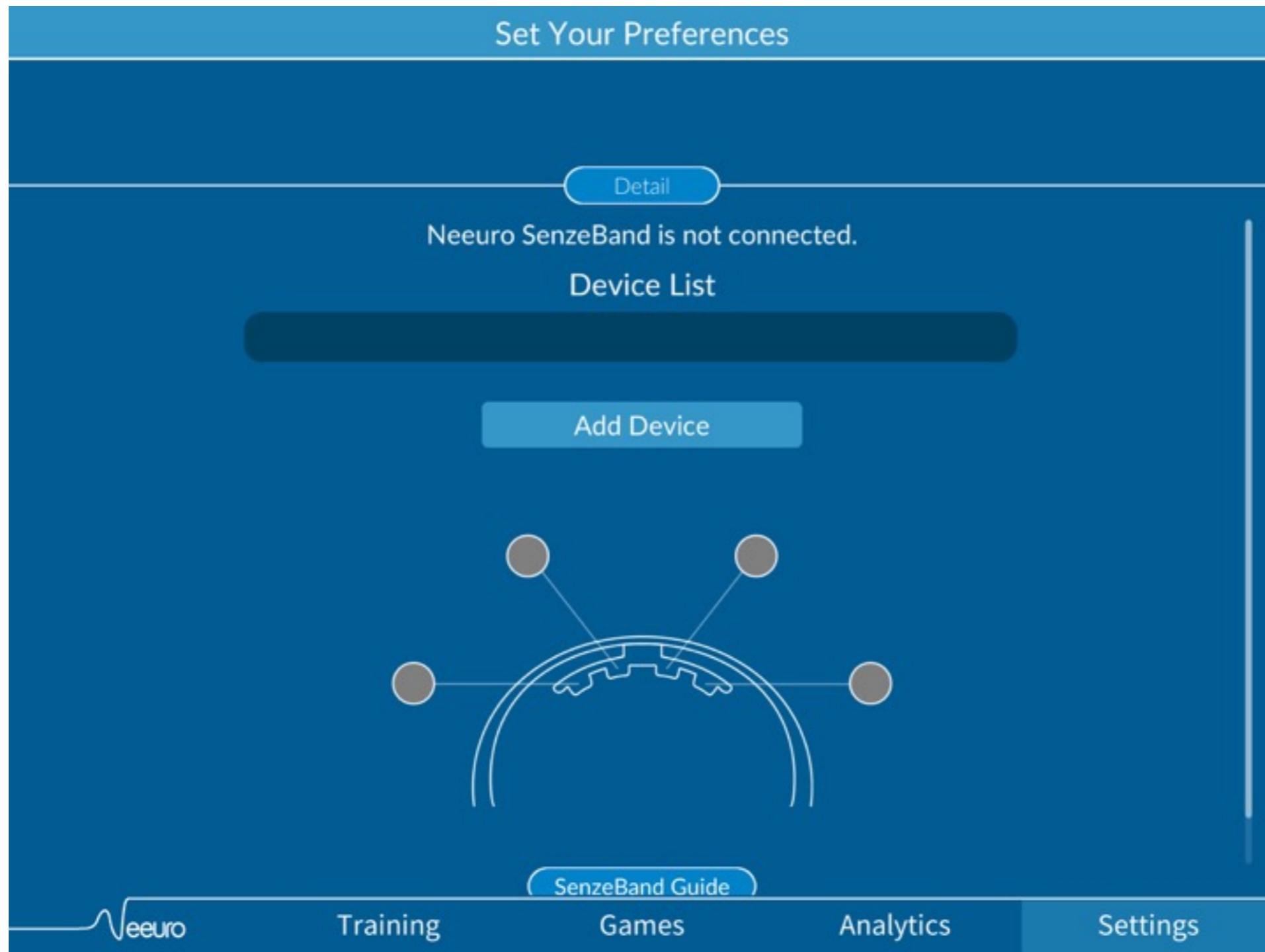

Tap on **Add Device**



New Connection to a Device



Memorie works best with Neeuro SenzeBand to give you an accurate measurement of your cognitive performance.

For more information, visit neeuro.com.

Next

Tap on **Next**



Power On the Device
Press and hold the power button for 2 seconds



Select a Device to connect to

Neeuro SenzeBand

Power on SenzeBand
Tap on **Neeuro SenzeBand** button when it appears



Power On the Device
Press and hold the power button for 2 seconds



If you are using a new SenzeBand, it will be activated and your account will be upgraded to Premium.
Premium account has access to all games and features in Memorie.
Would you like to connect to the SenzeBand?

Yes

No

Tap on **Yes**



New Connection to a Device



Verifying Connection...

Neeuro SenzeBand

Displays connection status
Please wait



New Connection to a Device




Connected

Neeuro SenzeBand 09381467

OK



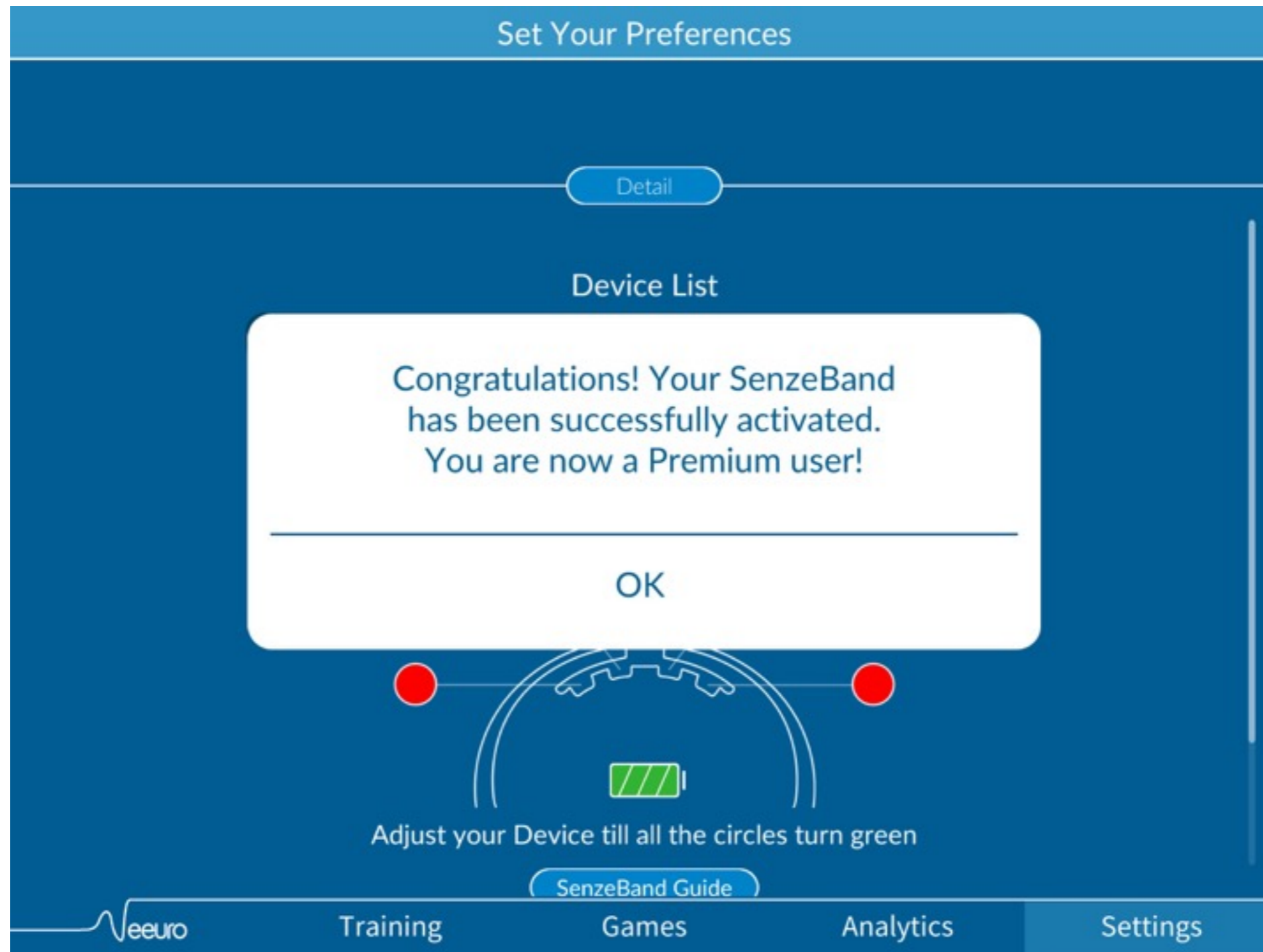



SenzeBand ID is obtained and displayed
Tap on **OK** when it appears




A confirmation message appears
Tap on **OK**,
Memorie app will logout and automatically re-login

Set Your Preferences

Detail


Account	Premium
<hr/>	
Email *	
<hr/>	
Name *	
<hr/>	
Change Password *	
<hr/>	
Retype Password *	
<hr/>	
Gender	
<hr/>	
Birth Date	 
<hr/>	

TrainingGamesAnalyticsSettings


Check that your Account has changed to **Premium**




Tap on **Device**, check that the SenzeBand appears on the Device List and is connected
Put on the SenzeBand, adjust for good contact and fit

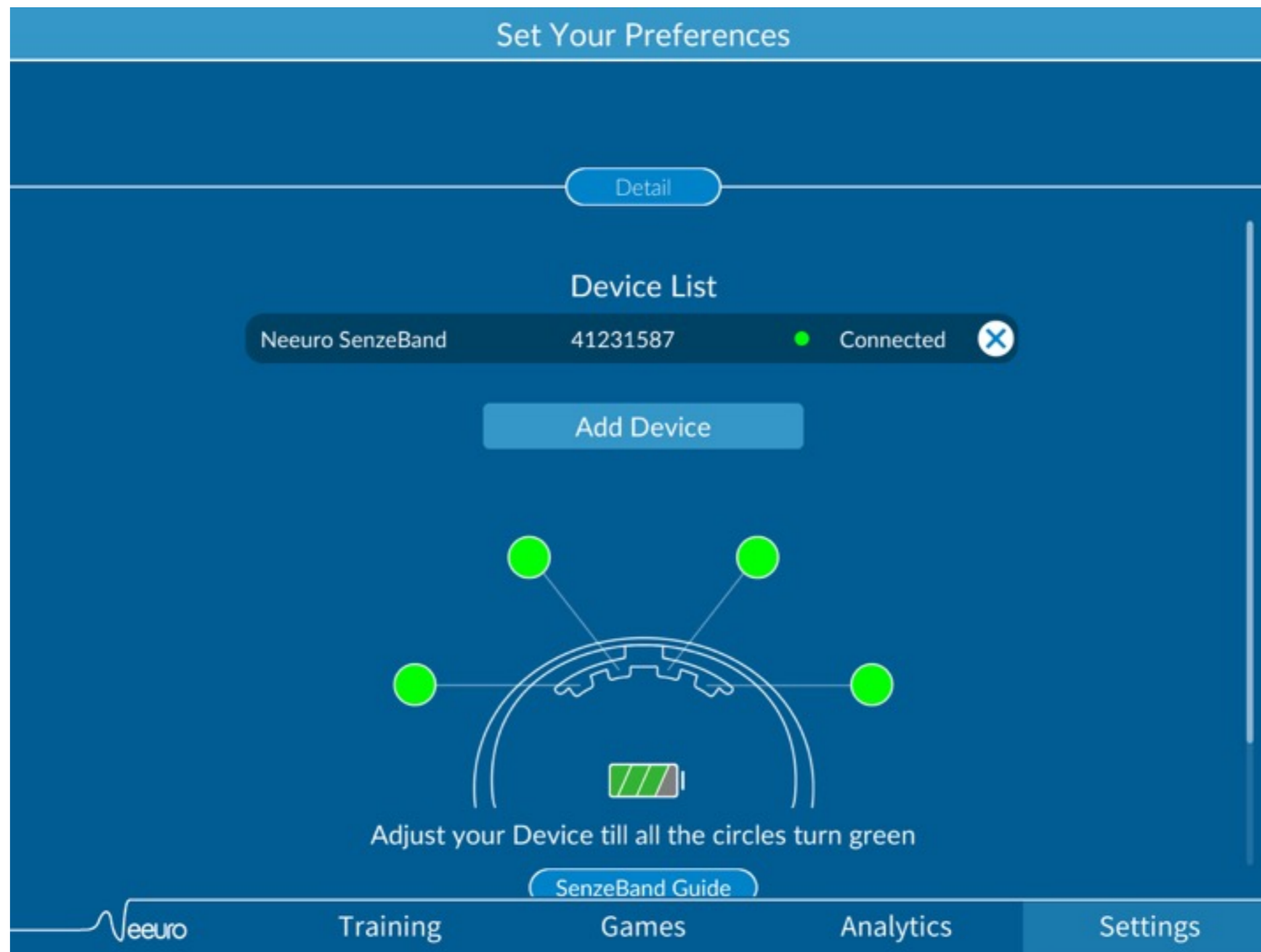

Adjust SenzeBand until indicators become **Green**

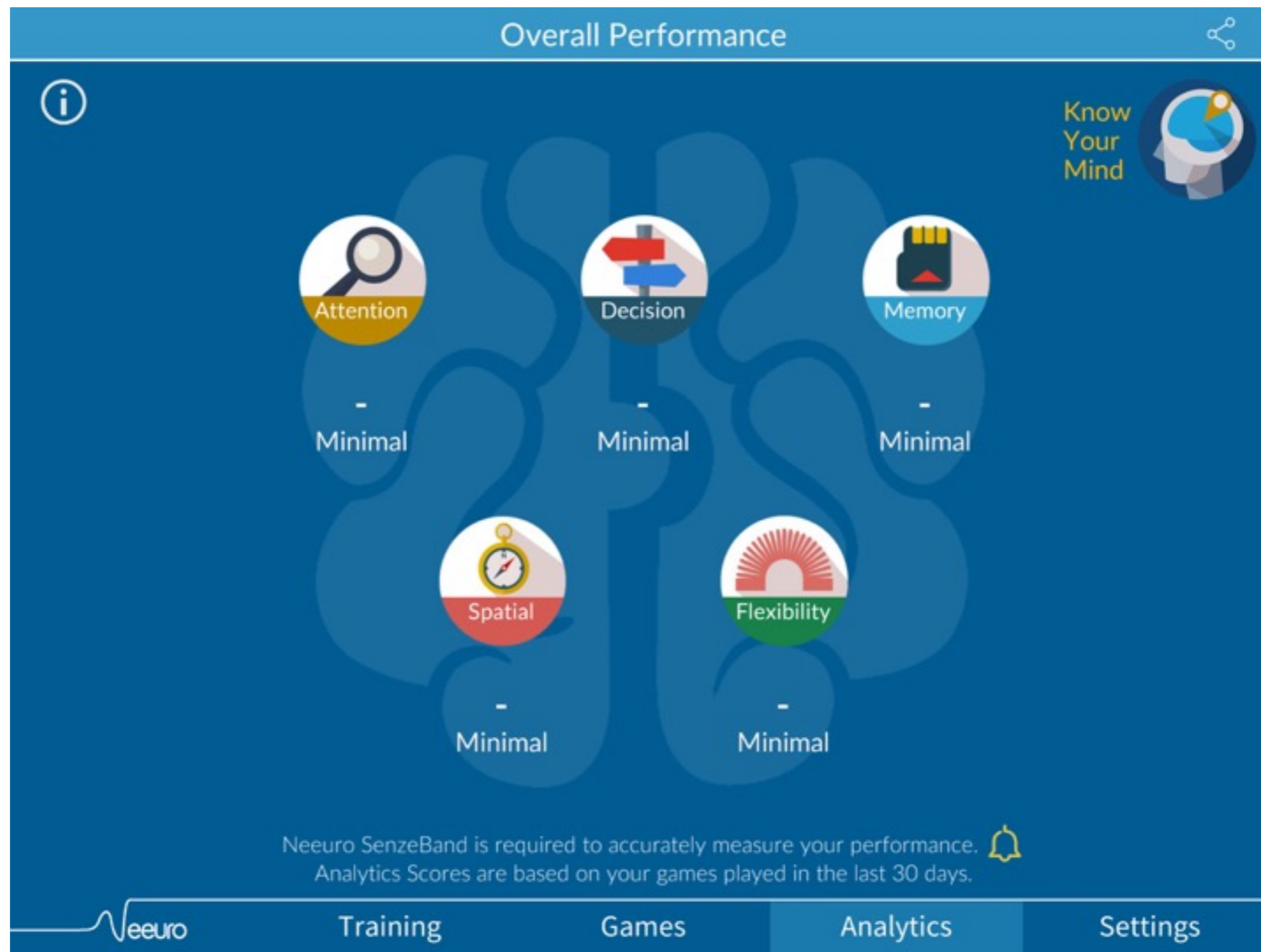

Tap on **Analytics-Know Your Mind**

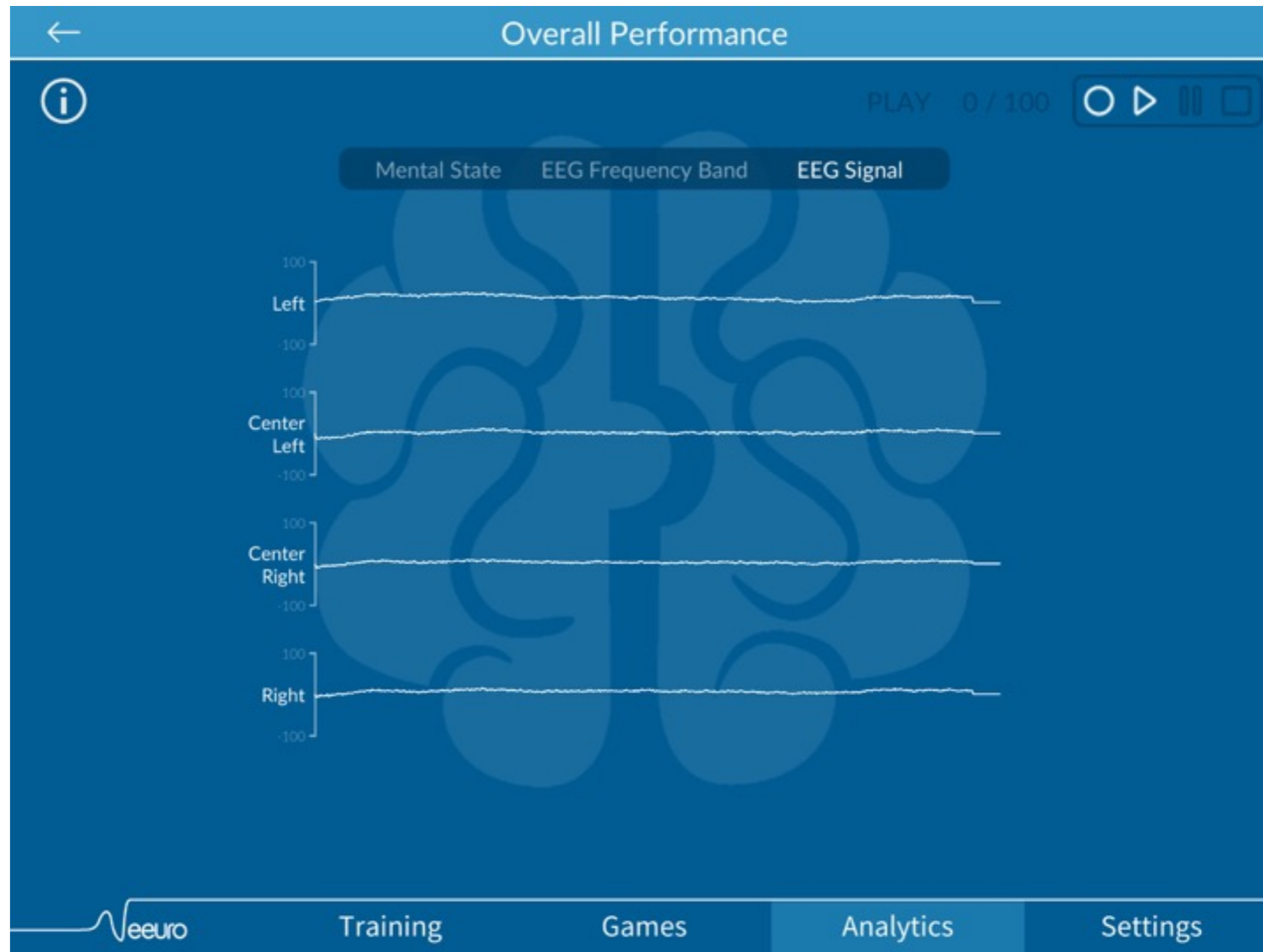

Observe live EEG signal from 4 sensors
You are ready to use your **SenzeBand**

Quick-start guide to SenzeBand

Memorie 1.1

Steps to Activate your SenzeBand

1. Search for “Neeuro Memorie” on Apple App Store or Google Play Store.
2. Install Memorie app.
3. Start Memorie app.
4. Enable Bluetooth, ensure Internet connection.
5. Sign up to get a Basic account.
6. In Memorie, login to your Basic account.
7. Go to “Settings”, “Device”.
8. Tap on “Add Device” button.
9. Power on SenzeBand.
10. In Memorie, search should show up a “Neeuro SenzeBand” button.
11. Tap on “Yes” to activate the SenzeBand.
12. App will automatically log-out and re-login.

13. Go to “Settings”, “Device”.
14. Tap on “Add Device”, search and connect to your SenzeBand.
15. “Neeuro SenzeBand” should appear on “Device List”.
16. When the 4 circle indicators turn green, the SenzeBand is connected properly, and the contact with the skin is good.
17. Go to “Games” and start your brain-training.

Steps to Connect SenzeBand to Memorie app

See above, steps 13 - 16.