



HOW TO USE TUCKY?

1) Charge Tucky with the USB cable

2 hours = 1 week of continuous usage



2) Press ON/OFF button to turn on Tucky

LED blinking for 2 seconds = ON


LED lighting during 2 seconds = OFF

3) Place the double-sided adhesive **A** on Tucky **B**

Peel adhesive and its clear film from paper backing

Place it on the front of Tucky

Remove both parts of the clear film

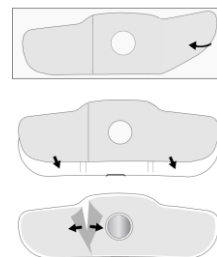


4) Stick Tucky under the armpit

Place it at the very top of the armpit


Wear clothes

~ 15 minutes is necessary to reach stable reading with closed arm

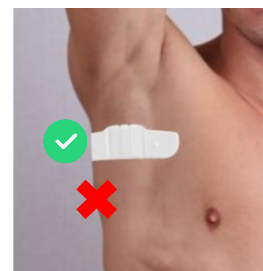

FOR BETTER HOLD ON ADULTS

- Shave the armpit
- Add a single-sided adhesive strip **C**

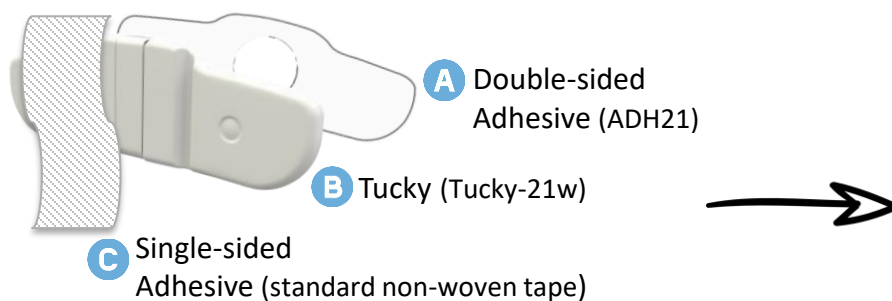

5) After use, clean the device with medical alcohol