QuietOn

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QuietOn

2 IMPORTANT INSTRUCTIONS FOR USE

How do I make sure my unit works and how do I use it correctly?

To help you get the most out of your QuietOn 3.1, we would like to give you a few tips to ensure that your device works properly.

When you remove the ear tips from the charging base, the device automatically switches to listening mode. Noise reduction is automatically activated after 20 seconds. This timer starts when the device is removed from the charging cradle.

If there is background noise, this change should be easily noticeable.

Proceed as follows

1. Move to the area where the noise is occurring.
2. Replace the ear tips and close the case.
3. Take both earplugs out of the case.
4. Insert the ear tips into your ears within 10 seconds.
5. Listen. You should hear outside sounds as you would in your normal environment.
6. After about 15 seconds, you should feel an immediate change and noise reduction is activated.

Please watch our video to learn the best way to use them:

<https://www.youtube.com/watch?v=LHMHQVdw3IU>

How can I ensure that my device is comfortable and pleasant to wear?

One of the most common complaints is that the earplugs don't sit deep enough in the ear. It may sound strange, but the deeper the device sits in the ear, the more comfortable it is. A deeper fit has two benefits:

1. firstly, the main part of the earplug does not stretch the auricle.
2. the product can rotate in the ear and put pressure on the ear canal if it does not sit deep enough.

To ensure that the earplug sits deep enough in the ear, we recommend using foam plugs that are small enough.

Here is a video to help you insert earplugs:

<https://www.youtube.com/watch?v=pcBkxHDUIml>

It is also important for comfort that the earplugs are not too large. If the earplugs have to be pushed back constantly while you are wearing them, it can cause pain in the ear. Please note that earplugs swell when they get wet. The swelling of the foam tips can cause additional pressure and discomfort in the ear. Make sure the tips and your ears are dry when using the product.

You can find more frequently asked questions and answers at this link:

<https://quieton.com/faq3/>

You are also welcome to contact us by telephone. We will be happy to advise you.

Kind regards Your MindTecStore Team

Service Hotline: 06403-60993-51 Email: info@mindtecstore.com